Hundepfote zum Krallenschneiden

Canine claw trimming paw

Bedienungsanleitung / User Manual

Art. Nr. / Item No. VET4750

Produktbeschreibung

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Product Description

The dog paw simulator is designed for learning and practicing the correct technique of cutting a dog's claws. The realistic silicone paw cast allows simulation of the procedure and related situations, such as potential complications like bleeding. The trainer enables users to learn about the anatomy of the claw and the differences between various types of canine claws.

Kit Contents

1. Stable base for securing the model
2. Silicone cast of a dog paw
3. Set of nails for installation:
 - Normal black (NB)
 - Normal clear (NC)
 - Long black (LB)
 - Long clear (LC)
4. Blood-mimicking fluid
5. Syringe for filling the „blood“ reservoir

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Educational Features:

- Training in the correct technique of dog claw cutting
- Learning about the different types of dog claws
- Realistic discharge of „blood“ when the procedure is not done correctly
- Learning how to react to bleeding
- Ability to learn how to bandage a dog's paw

Preparation for Use

1. Place the trainer on a stable, flat surface.
2. Attach the chosen claws to the paw cast according to the assembly instructions.
3. Using the syringe, fill the blood-mimicking fluid system with a small amount of the provided liquid.
4. Protect the area around the simulator in case of fluid leakage during training.
5. Prepare appropriate claw-cutting tools and bandaging materials if first-aid training is planned.

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Training Instructions

1. Ensure all components are properly installed and the trainer is ready for use.
2. Hold the paw securely with one hand to stabilize it.
3. Using the other hand, carefully trim the claws following the recommended technique.
4. Take special care not to damage the claws quick, especially when working with dark-colored claws – the realistic model allows for practicing identification and avoiding this sensitive area.
5. If „blood“ appears, apply hemostatic techniques and dress the wound.
6. After your training session, empty and wash the silicone elements and allow them to dry thoroughly.

Cleaning and Maintenance

- After each use, empty the blood-mimicking fluid reservoir.
- All silicone parts should be hand-washed with warm water; do not use abrasive cleaners.
- Store dried elements in a dry place, away from heat sources and sunlight.

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Warnings and Safety

- Trainer intended for educational use only.
- Do not use real tissues, biological fluids or sharp tools beyond standard nail clippers.
- The trainer should always be used under adult or instructor supervision.

EXPERTS IN MEDICAL EDUCATION

Erler-Zimmer GmbH & Co.KG

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T +49 (0)7841 / 67191-0 · F (0)7841 / 67191-99

info@erler-zimmer.de

www.erler-zimmer.de

Follow us!