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Male Catheterization and Enema Simulator

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! Handling and Safety Precautions:

These precautions should be strictly observed in order to ensure safe, long-term use of the product.

1. This simulator is made to have the same feel as that of a living body. Undue handling may cause breakage of the simulator.
2. The simulator reproduces the resistance of the bulbar urethra. When inserting the catheter, if the penis is lowered to an angle less than about 60 degrees, the insertion of the catheter may be difficult. Accordingly, please hold the penis at an angle of about 60 degrees.
3. When practicing cleaning or disinfecting of the pubic area, please use water. Use of alcohol may cause breakage of the simulator. Use of colored disinfectant liquid may cause pigment to be fixed to the simulator so that the pigment cannot be removed.
4. As for the lubricant, please use waterbased lubricant.

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5. Please do not use oily lubricant (olive oil etc.), alcohol lubricant, and lubricant containing organic solvent. These may cause breakage of the simulator.
6. Please use catheters of 14Fr and 16Fr.
7. Please check the state of the catheter before inserting the catheter. Please do not use a catheter that is degraded or deformed. Such a catheter is difficult to insert.
8. When inserting the catheter, please apply lubricant each time.
9. If it is difficult to insert the catheter, please increase the amount of the lubricant or use another catheter. The simulator may break if a catheter is forced to be inserted.
10. When indwelling the catheter, please inflate the balloon after inserting the catheter apart from the balloon end portion by a distance equal to or greater than 23 cm. Indwelling of the catheter in an intermediate of the valve causes the valve and the catheter to be broken.

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11. If the insertion of the catheter is repeated, water may accumulate in the urethral canal. If the simulator is used in such a state, water may leak through the urethral exit. Please slant the simulator suitably to drain the water.
12. After use, please wash the simulator and catheter well with water. If lubricant dries and hardens, it may interfere with the use of the simulator.
13. Please check the state of the valve at every use. If the valve is broken, it may cause a leakage of water.
14. Oil components may come out of the genital. After use, please do not keep the genital attached to the simulator. Please wrap it in cooking paper, gauze, etc., put it into an attached reclosable poly bag, and keep it. For a detailed storage method, please refer to the cleaning and storage methods on page 26.
15. Please check the state of the genital every three years as a reference, and exchange it accordingly.
16. Please do not place objects on the simulator. Otherwise, the simulator may be deformed.

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17. Please do not keep the simulator in contact with printed material as newspapers or copies. Print may transfer to the simulator and be fixed there.
18. When storing the simulator, please select a place that is not subject to direct sunlight and ultraviolet rays.

1. The contents of this instruction manual are subject to change without notice.
2. This instruction manual may not be reproduced in part or in its entirety without permission.
3. Please contact the manufacturer in the event that any errors or omissions are found in the contents of this instruction manual.
4. This product should be used only as described in this instruction manual. In particular, the product should not be used in any way that contravenes the precautions noted in the instruction manual.

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Table of Contents

1. Outline and Features
2. Components and Configuration
3. Handling
4. Specifications
5. Replacement Parts
(Sold Separately)
6. Cleaning and
Storage Methods

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1. Outline and Features

Outline

The simulator imitates the shape of the urethral canal peculiar to a male. A catheterization catheter is inserted according to a suitable technique to reach the bladder. This causes urine (water) to flow out. The feel of the penis is closely similar to that of a living body penis. Techniques such as disinfecting of surroundings of the urethral exit and cleaning of the pubic area can be performed. The pelvis can be separated at the middle so the shape of the urethral canal, the insertion state of the catheter, and the anatomical positional relationship can be controlled.

Features

- The simulator is of an actual size by which a catheterization and glycerine enema for an adult male can be practiced.
- Since soft and stretchable materials are used for the pubic area, the appearance and the feel are realistic to give presence when practicing.
- A series of techniques of catheterization, such as disinfecting of the pubic area, inserting, fixing, and removing of a catheter, can be practiced.

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- The simulator is shaped to imitate the bending of the scrotal-angled portion of the urethral canal and the bulbar urethra. The simulator is configured such that it is difficult to insert the catheter if the penis is not appropriately raised.
- Techniques, such as retracting the foreskin, disinfecting and cleaning, can be performed.
- The motion of the penis is also close to that of a living body penis. This enables the practice of fixing the penis.
- The pelvis can be separated, and the urethral canal and bladder are transparent. This enables checking the insertion state of the catheter into the urethral canal and the balloon in the bladder.
- By exchanging the prostate, the urethral stricture due to hypertrophy can be reproduced
- The attached a glycerine enema-tor can be inserted through the anus to the rectum. This enables the practice for the glycerine enema.

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

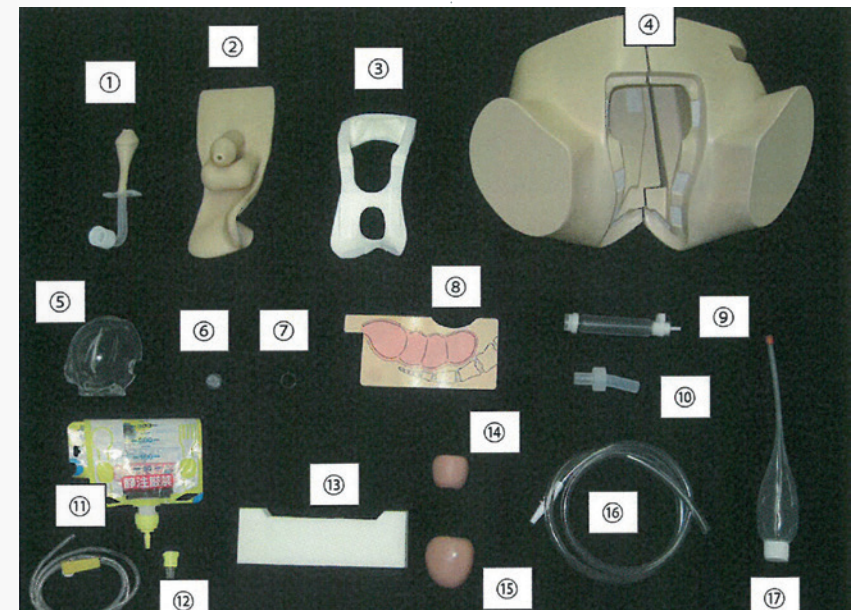
Storage bag
Instructions manual (this pamphlet)

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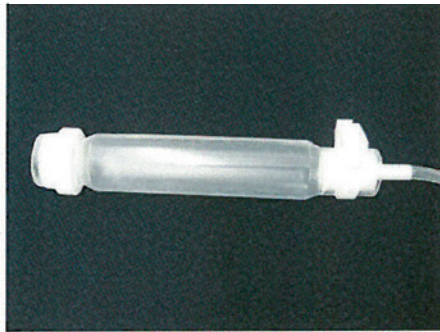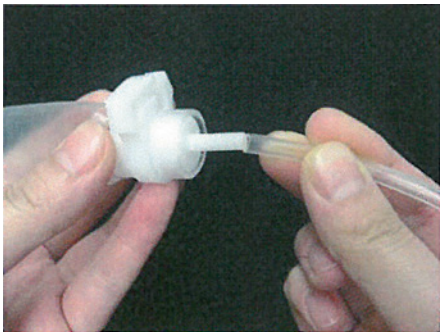3. Handling the Model

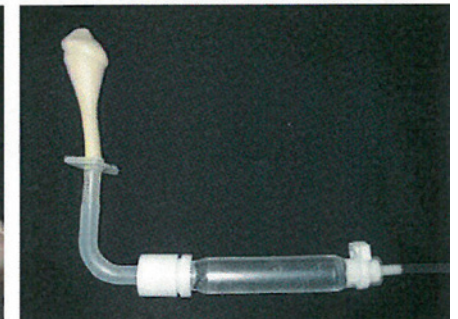
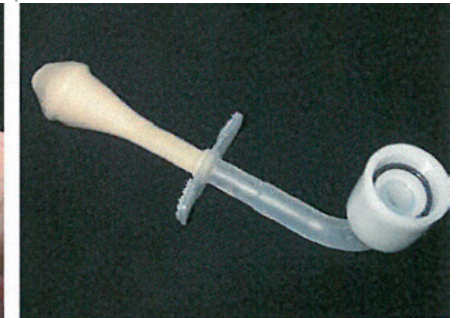
3-1 Assembly of simulator

1. Please connect the bag with the bladder tube, and put water into the bag. Please open a clamp, fill the bladder tube with water, close the clamp, and attach the bladder valve to it.

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
2. Please attach the O-ring to a connector of a urethral part, and connect it to the bladder tube.

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
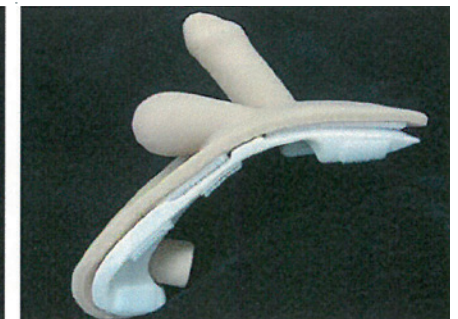
3. Please attach the genital to the pubic area frame.

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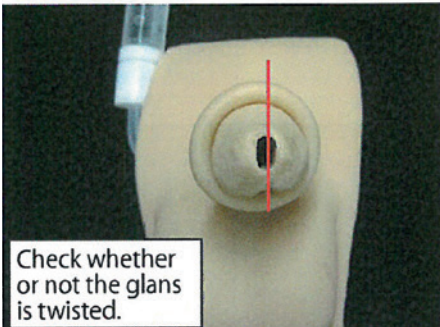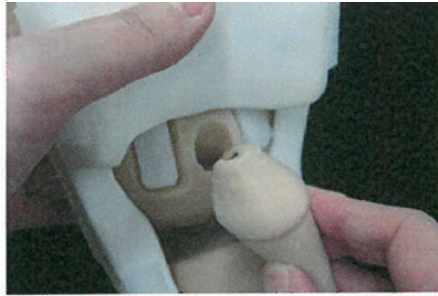4. Please apply lubricant to the distal end of the glans and insert it into the genital. Please check whether or not the glans is twisted and then fix the urethral part and the genital with hook and loop fasteners.


Check whether or not the glans is twisted.

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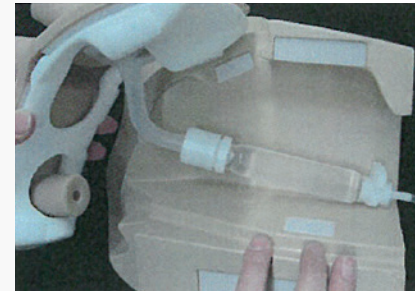5. Please separate the pelvis into right and left parts, and fit and fix the pubic area frame to the the right part of the pelvis.

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

6. Please attach the bladder to the bladder tube, and fit and fix the bladder to the hook and loop fasteners of the right part of the pelvis.

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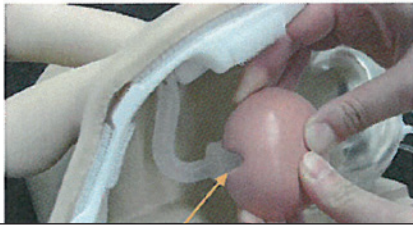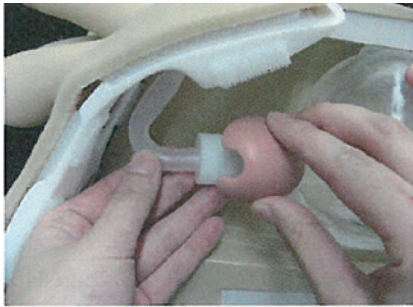
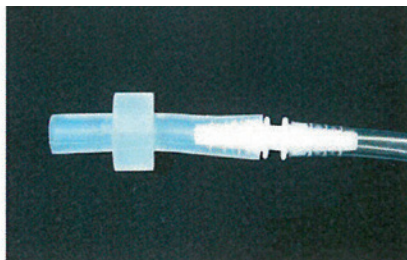
7. Please cover the connector for connecting the urethral part and the bladder tube with the prostate. When covering the connector with the prostate (hypertrophy), please stretch and extend a constricted portion of the prostate by hand, and attach the prostate such that the prostate clamps the connector.

8. Please connect the rectum connecting connector with the drain tube.

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
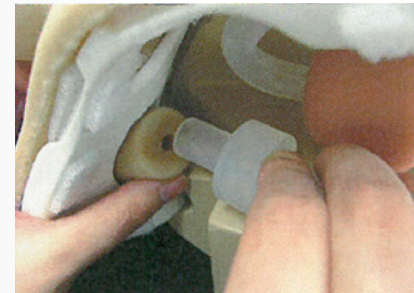
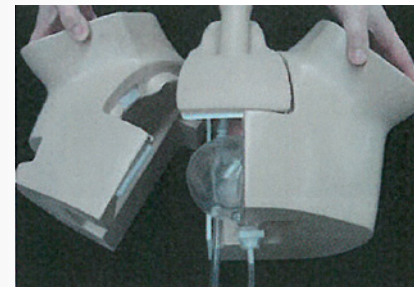
9. Please insert the rectum connecting connector from the rear side of the genital.

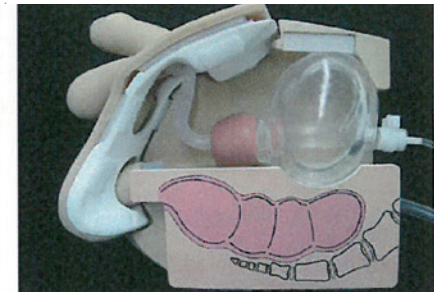
10. Please fit and fix the rectal plate to the right part of the pelvis.

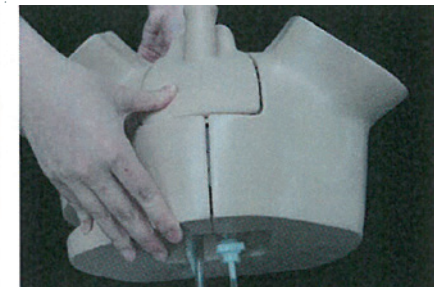11. Please attach the right and left parts to close and fix the pelvis, and check whether the pubic area frame is fixed to the pelvis.

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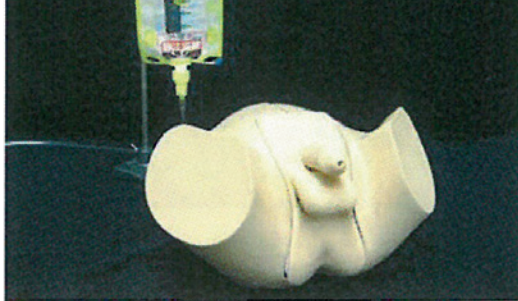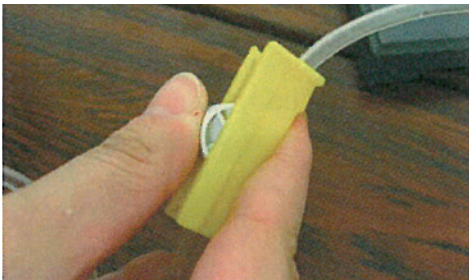
12. When performing a catheterization, please hang the bag for irrigator at a position higher than that of the simulator pelvis. Water does not flow out.

13. When performing an enema, please prepare a bucket etc. under a desk or a bed and put the distal end of the drain tube in it.

3-2 Insertion of Catheterization Catheter

1. Please open the clamp of the tube for irrigator and check whether or not the water leaks from the simulator.

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
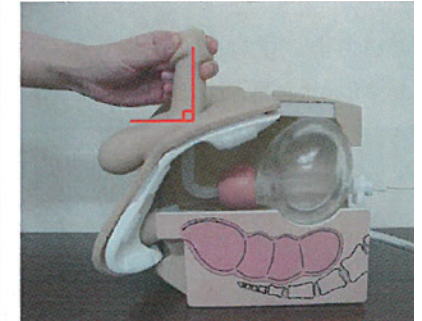
2. Please lower the foreskin, expose the glans, and erect the penis at about 90 degrees. ***When practicing the cleaning or disinfecting of the pubic area, please use water as the disinfection liquid.**

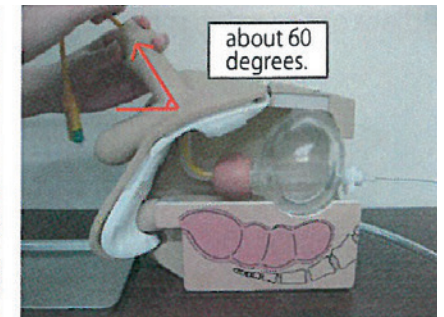3. Please apply lubricant on the distal end of the catheter for about 4 to 5 cm and insert the catheter. If you feel resistance, please slant the penis forward at about 60 degrees, and advance the catheter such that you slightly draw the penis to yourself.

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* Intermediate resistance reproduces resistance of the bulbar urethra.

* The simulator is configured such that it is difficult to insert the catheter if the penis is lowered to an angle less than about 60 degrees.

* If it is difficult to insert the catheter, please check the techniques and insert it again. In this case, please remove moisture from the catheter and increase the amount of the lubricant to be applied. Also, please consider using other types of catheters. The simulator may be broken if the catheter is forced to be inserted.

* If it is difficult to insert the catheter, please use water mixed with a small amount (approx 5 %) of surfactant (neutral detergent) as urine.

* If the insertion of the catheter is repeated several times, water may accumulate in the urethral canal. If the simulator is used in such a state, water may leak through the urethral exit. Please slant the simulator suitably to drain the water through the external urethral exit.

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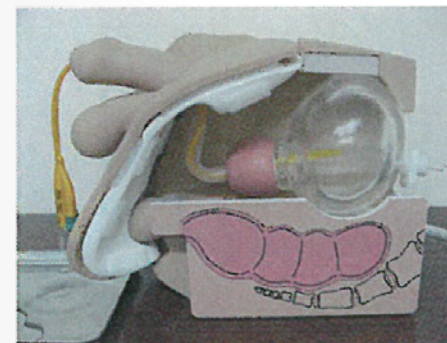
4. Please check the outflow of the urine (water). Please check the following if the urine does not flow out.

- Does the catheter pass through the valve?

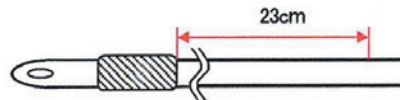
- Is the clamp of the tube for irrigator open?

- Has the amount of the water in the bag for irrigator decreased to a small amount?

5. When you check the insertion state of the catheter or the indwelling state of the balloon, please separate the pelvis into the right and left parts to be used. In this case, please check whether the pubic area frame is firmly fixed to the right part of the pelvis.

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When indwelling the catheter, please inflate the balloon after inserting the catheter apart from the balloon end portion by a distance equal to or greater than 23 cm. Inflating the balloon in an intermediate of the valve may cause the valve and the catheter to be broken.

6. Once the catheter is inserted, please remove moisture, apply lubricant each time, and then insert the catheter. If the lubricant is applied where water remains, lubricant slides easily off of the catheter. This makes it difficult to insert the catheter.

3-3 Enema

1. Please prepare a bucket etc. under a desk or a bed and put the distal end of the drain tube in it.
2. Please insert the stand in the groove of the left part of the pelvis, and fix the pelvis in the left lateral decubitus.

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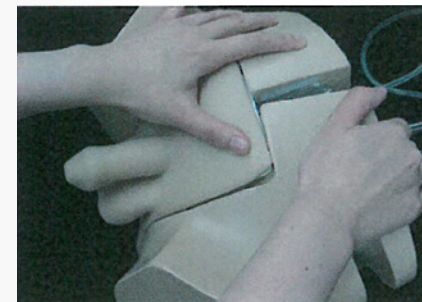
3. Please apply lubricant to the distal end of a enema, insert it into the anus, and pour enema liquid (water) into it. At this time, please lower the distal end of the drain tube to a position below the anus. Otherwise, the enema liquid (water) may flow backward.

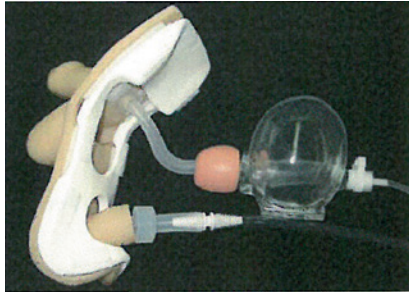
3-4 Removal

1. Please separate the pelvis and remove the rectal plate.

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
2. Please remove the pubic area frame from the pelvis, and remove the prostate, the bladder, and the rectum connecting connector.
* Please pay attention to the outflow of the water.

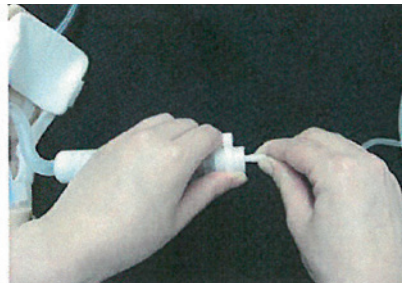3. Please close the clamp of the tube and remove the bladder tube.

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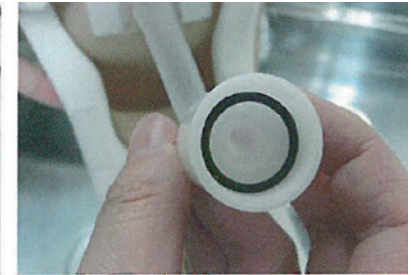
4. Please remove the connector for connecting the bladder tube with the urethral part.
* At this time, please pay attention to a loss of the O-ring.


5. Please remove the bladder valve from the bladder tube, and drain the water in the tube.

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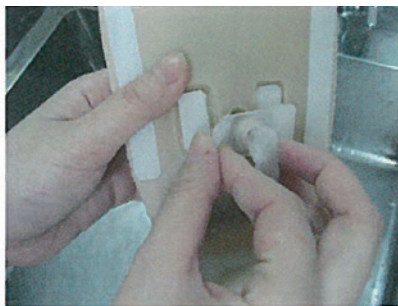6. Please remove the genital from the frame, and remove the urethral part from the genital.

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
Product	Length	Width	Height	Weight
Pelvis	Approx. 25 cm	Approx. 39 cm	Approx. 21cm	Approx. 2,3 kg
Storage Bag	Approx. 29 cm	Approx. 53 cm	Approx. 32 cm	---

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Product	Itemno.
Genital skin	LM-109MB
Bladder Valve	LM-109V
Bag for irrigator (3pcs/set)	LM-025J
Tube for irrigator (3pcs/set)	LM-025H
Glycerine enemator (with cap/5pcs)	LM-0686

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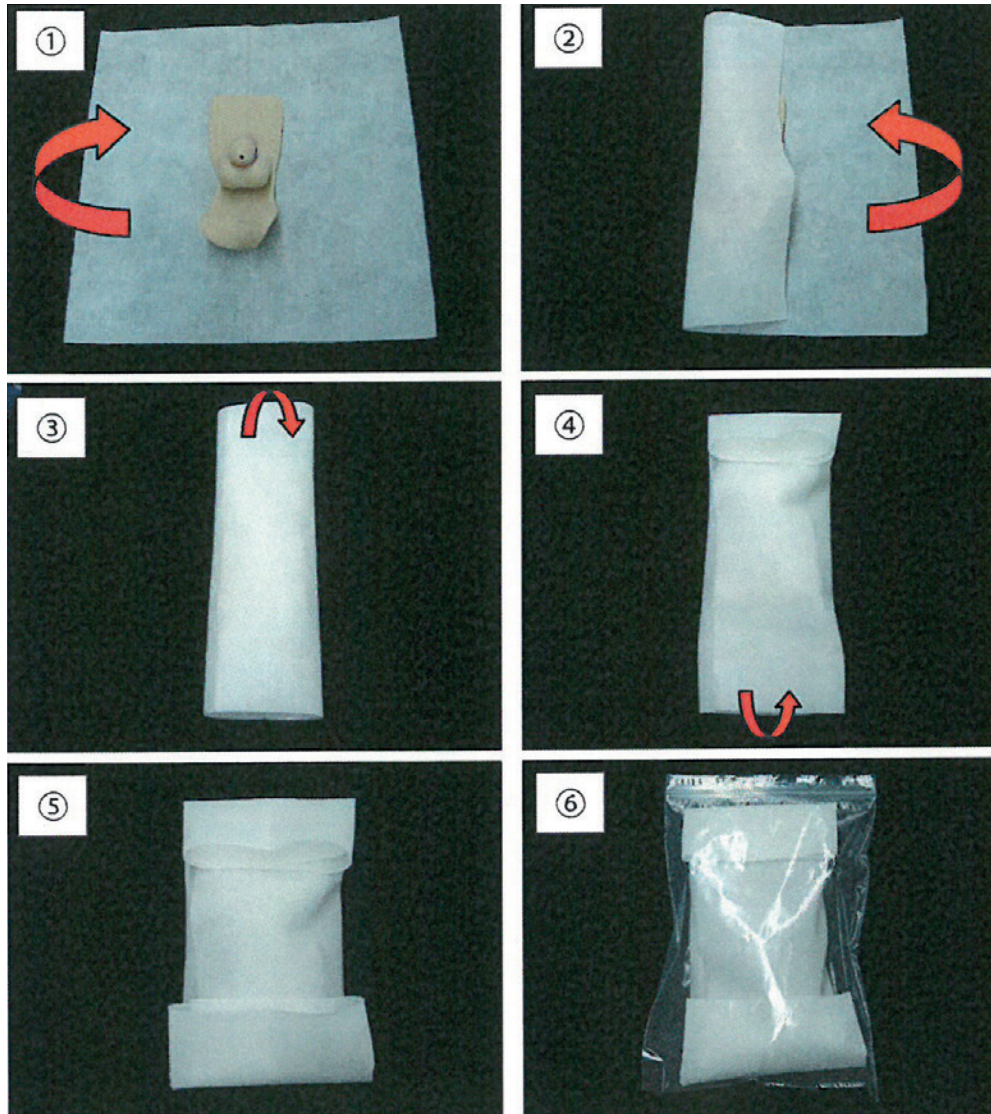1. Please flush the lubricant that has adhered to the urethral part, the bladder valve, the bladder tube, etc. well with water. If the lubricant dries and hardens, it may interfere with the use of the simulator.

2. Please remove moisture well, dry the simulator and then put it into a storage bag.

3. Oil components may come out of the genital. After using the simulator, please do not keep the genital attached to the pelvis. Please wrap it in cooking paper, gauze, etc., put it into an attached reclosable poly bag, and keep it therein. Please wrap the genital gently such that wrinkles are not generated on it.

4. Please apply baby powder that is commercially available, if you mind stickiness of the genital.

5. As for stains on the simulator, please use neutral detergent that is watered down and wipe off the stains with wet gauze etc.

*** Please do not use solvents such as thinner and benzene.**

EXPERTS IN MEDICAL EDUCATION

Erler-Zimmer Medical GmbH

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T +49 (0)7841 / 67191-0 · F (0)7841 / 67191-99

info@erler-zimmer.de

www.erler-zimmer.de

Follow us!