# User guide

## Diabetes Belly

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### **Thank you for purchasing this practice model for abdominal injections.**

It helps you to practise the correct injection of insulin. The material can be formed into an abdominal fold so that you can practise injecting with a syringe or pen. The injection of liquid is possible, we recommend distilled water. When injecting real medication, there is a risk of the medication reacting with the plastic and causing damage. As the chemical composition of the medication can vary, we cannot accept any liability for damage caused by injected medication.

Care for the model after each use. Wipe the surface with a damp cloth and then dry it. Rub the surface from time to time with a little talcum powder or baby powder to keep it soft and smooth. After using liquid, remove the protective plate and the foam, squeeze out the foam and allow both to air dry. When everything is dry, reassemble it.

The use of thin needles extends the life of the model. Do not use any printed paper or objects with the model, as the printing ink can penetrate the plastic and cause indelible stains.

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*Experts in medical education  
since 1950*

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### **Erler-Zimmer GmbH & Co.KG**