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Art. Nr. R19100

CPR Training manikin

Item no. R19100

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Art. Nr.: R19100I

Content

Check out the contents of the box. It should contain: one manikin torso, one manikin head, 10 Face Shield Lung Bags and a nylon carrying case. If you are missing anything, please contact your Distributor. All components, including the Filtered Face Shield and the Face Shield Lung Bag, are latex free.


Head


**Carrying Case
Single manikin**


**Carrying Case Four
Pack Manikins**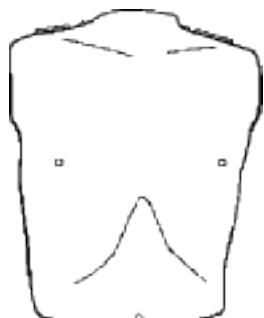

Torso

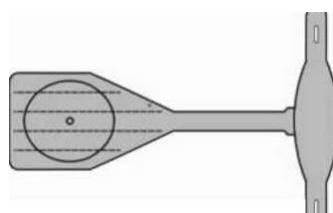Face Shield

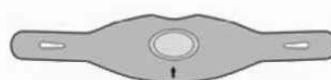**Filtered Face Shields
(not included - available
seperately)**

- The face and the body skin of the CPR Manikin is made out of the same material and can be cleaned in the same manner
- If the material becomes dirty, clean with a mixture of soap and water. If the area is extremely dirty, common household cleaners such as Formula 409, Windex or Vim may be used. It is recommended that the use of these cleaners be done cautiously with minimum amount of abrasion to insure the life of the skin.
- The face skin many also be sanitized with alcohol wipes or soaked in a solution of 1/4 cup bleach mixed with one gallon of water for 10 minutes. Between student uses, the instructor (or student) may clean the faces off with an alcohol wipe as an extra measure of safety.
- Don't forget to wipe the upper chest foam & skin in the same way in order to sanitize the in side of the manikin.

CPR Rate Monitor Guide

The CPR Training Manikin Rate Monitor has a unique way of monitoring both the depth of compressions and the rate of compressions.

At the first compression all four of the lights in the upper left shoulder will come on, indicating that the lights are working and the batteries are good. (Figure #1)

If the lights do not light, make sure that the depth of compression is deep enough (you will hear a clicking sound). The lights will not work if the student is not compressing the manikin correctly.

If the depth of compression is correct, and the lights do not come on, replace the two AA alkaline batteries in the battery compartment located in the back of the manikin in the left shoulder.

Once a student begins chest compressions, the amber light and the green lights will go out and, from zero to less than 60 compressions per minute, the red light will be on. (Figure #2)

As the student increases the rate of compressions and reaches 60 compressions per minute, the red light will go off and the amber light will come on until the student reaches 80 compressions per minute. (Figure #3)

As the student increases the rate of compressions and reaches 80 compressions per minute, the amber light will go off and first green light will come on until the student reaches 100 compressions per minute. (Figure #4)

When the student reaches 100 compressions per minute, the second green light comes on, indicating that the proper rate as been attained. (Figure #5)

If the student slows down, one of the green lights will go out indicating that the students needs to increase the rate of compressions.

If the student's depth of compression is not correct, at any time, all the lights will go off and the red light will blink.

Figure 1

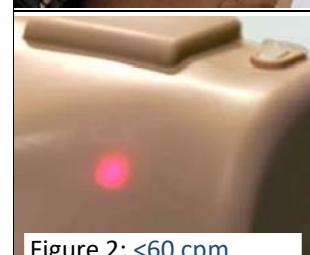
Figure 2; <60 cpm

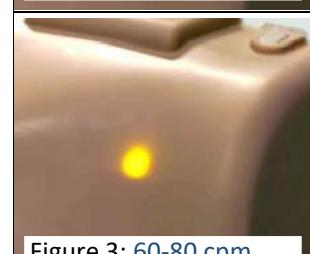
Figure 3; 60-80 cpm

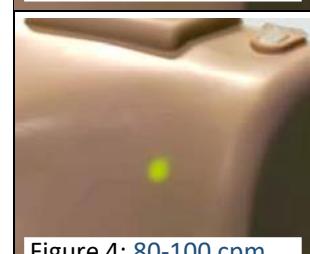
Figure 4; 80-100 cpm

Figure 5; 100-120 cpm

Face-Shield/Lung-Bag Installation

1. Open the torso of the CPR Manikin by pulling the ,tabs' located on the shoulders of the CPR Manikin and lift up the torso skin (Figure 1).

Figure 1

2. Attach the head to the torso by placing the neck of the head onto the two spindles on the neck portion

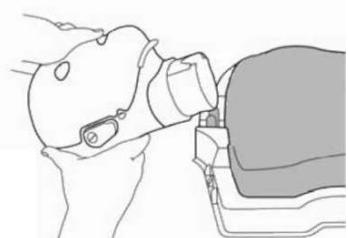
Figure 2

3. Fold the face shield lung bag:

- The face shield lung bag has four ,dotted lines' printed on one side of the bag (along with an oval)
- Using the four dotted lines, fold the face shield lung bag bellow into fourths, following the diagram below

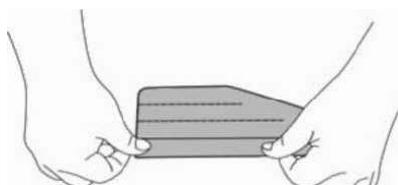
Figure 3a

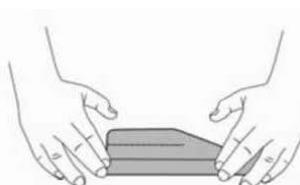
Figure 3b

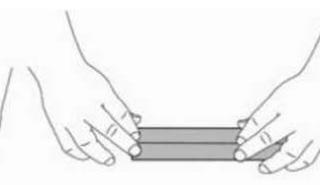
Figure 3c

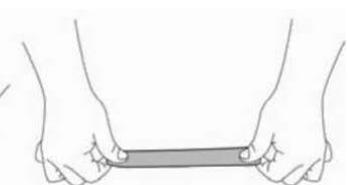
Figure 3d

4.Tilt the head of the manikin (Figure 4).

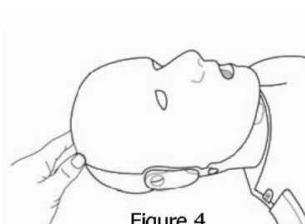
Figure 4

5.Insert the lung bag, folded end, into the mouth of the manikin (Figure 5).


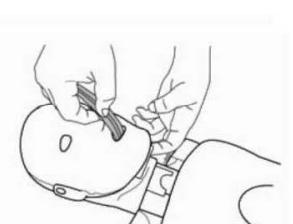
Figure 5

6.Push the lung bag into the mouth until the bag exits through the neck (Figure 6).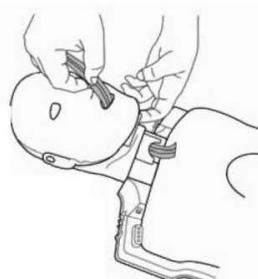

7.Grasp the lung bag and pull it through the neck (Figure 7)

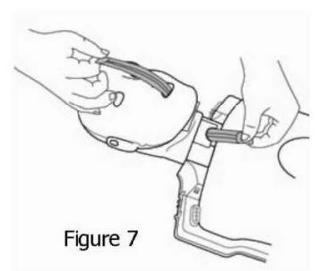
Figure 7

8.The face shield lung bag is then placed on top of the compression plate (the hard plastic piece found on top of the torso foam core) with the oval printed on the face shield lung bag matching the compression plate (Figure 8).

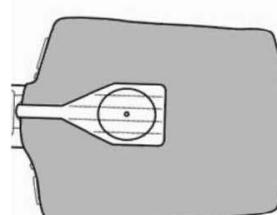
Figure 8

9.MAKE CERTAIN THAT THERE ARE NO TWISTS IN THE LUNG BAG!

Figure 9

10. Attach the face shield tabs to the ,buttons' located on the side of the manikin head (Figure 9).

11.The torso may then be closed by pushing down on the shoulder tabs until they snap into place.

Troubleshooting

- **If you cannot get the chest to rise when you blow into the face shield lung bag:**

- Check to make sure that the head is tilted back
- Check the lung bag itself. Insure that it is properly positioned over the compression plate, not twisted and is it laying flat?
- Check the face shield. Insure that it is attached to the 'buttons' on the side of the manikin head and that it is flat against the face of the manikin
- Check the bag to insure that has not been torn

- **If you do not hear a click when you compress the chest of the manikin**

- Make sure that you are pressing the compression plate far enough down so that you hear the click when you push the plate down about 1.75"

- **If you do not see the lights light during compressions**

- Check to see if there are two double 'A' (AA) batteries installed in the battery compartment behind the left shoulder of the manikin
- Insure the batteries that are installed are not dead

Consumables

Sanitary Airways for R19100
Pack of 50 pieces

Item no.: R19100A

Face shields for R19100
Pack of 50 pieces

Item no.: R19100B

Spare parts

Bag for R19100
1 piece

Item no.: R19100C


Complete replacement torso with indicator light for R19100

Item no.: R19100J

Clicker for R19100 (Without pressure plate)

Item no.: R19100E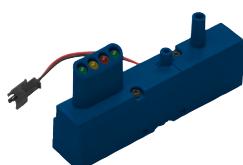

Indicator light for R19100 with AA batteries

Item no.: R19100F

Replacement face skins for R19100
Set of 4 pieces

Item no.: R19100G

Replacement torso skins for R19100
Set of 4 pieces

Item no.: R19100H

Complete replacement head for R19100

Item no.: R19100I

Interested in more?

Prestan AED UltraTrainer

Item no. R19550

The Prestan AED UltraTrainer™ offers features not found on any other universal AED trainer and is the most accurate representation of how current AEDs in the marketplace perform. This compact and lightweight unit will give students a realistic training experience with training pads that sense when they are placed on a manikin. In addition, the pads are preconnected and voice prompts are clear and calm.

Instructors can customize their training classes with options to turn on or off the compression metronome and/or the "give breaths" voice prompt.

With a 3-year warranty and durable long-lasting pads, the AED UltraTrainer was built to last.

Prestan is the First and Only Universal AED Trainer with:

- Dual Graphic (Adult/Child) Training Pads
- Pad Sensing System
- Preconnected Pads
- Child Training Button
- Child AED specific voice prompts
- Automatic or Semi-Automatic shock simulation
- CPR Coaching, "Give Breaths" voice prompt
- Turn on or off compression metronome
- Turn on or off "Give Breaths" prompts
- Low Battery warning
- 3 Year Warranty

Other features:

- Two languages
- 2015 Guidelines Compliant and Upgradable
- Play/Pause Button
- 5 Training Scenarios