

# CPR Training dog

Item no.: VET4660

*Cooper*



*Experts in medical education  
since 1950*



**ERLER ZIMMER**

Erlar-Zimmer GmbH & Co. KG · Hauptstr. 27 · D-77886 Lauf  
Tel.: (+49) 7841 / 6003-0 · info@erler-zimmer.de · www.erler-zimmer.de



◀ The model consists of 4 parts:

The body, sponge block, airway and pulse pump.

## Insertion & removal of the airway



1. Remove the sponge block and pass the airway bag through the opening in the neck area to the nostril.

Insert the airway with the open side first.



2. Grab the airway at the nostril opening and pull it approximately 10cm out of the opening. Now place the open side of the airway over the dog's nose.

3. Lay the bag flat inside the inner part




4. Important! The tongue must be pulled out halfway, otherwise ventilation will not work.

**5. Removing the airway:**  
Important! DO NOT pull the bag out through the nose, pull it out at the opening in the neck area.



**6. Pulse control:**  
Connect the manual pulse pump to the hose located on the hind leg.


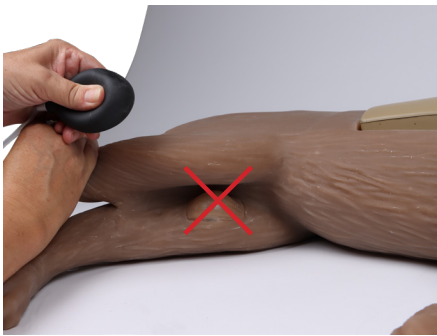


**7. ATTENTION! The pulse pump may not be pressed too hard!**

Just press the pulse pump gently to demonstrate the pulse rate.



**8. further information:**  
The dog's tongue can be completely pulled out and reinserted.

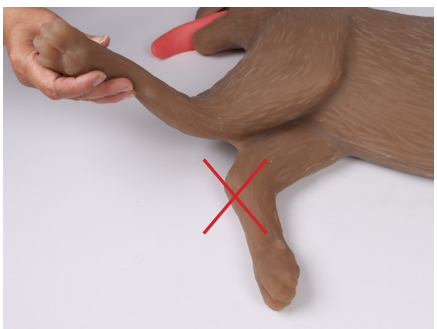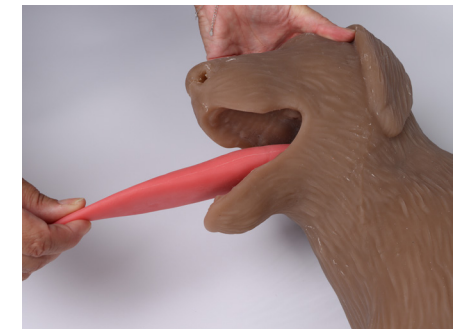

**9. Danger of tearing**  
The practice model is made of soft material. This allows the head to hyperextend and rotate.  
**Please don't:**  
-pull the front and hind legs apart.  
-Do not carry the dog by its limbs. You can see how to carry it correctly in the picture.

