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## Product Description

The simulator is designed for educational and training purposes in the correct technique of catheterization of female dog. The device realistically simulates the anatomical structure of the urogenital tract, including the vagina, urethra, and bladder. The simulator shows the back part of a medium-sized dog breed. The dimensions of the simulator are about 23 cm high, 27 cm wide and 31 cm deep.

### Product Description

- Silicone body with built-in fluid reservoir
- Silicone insert simulating vagina and urethra
- Artificial bladder
- Urinary adapter
- Silicone anus insert
- Base for the body

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## Preparation for Use

1. Place the simulator on a stable surface.
2. Insert the anatomical inserts (vaginal-urethral and anal) into their designated openings (vaginal at bottom, anal at top)
3. From the inside of the simulator, attach an adapter to the open tube of the vaginal insert. Make sure the adapter is properly attached. Then connect the lower end of the adapter to the body, and to the last free exit of the adapter connect the silicone pear that functions as a urinary bladder.
4. Check all connections: ensure the bladder, tubing, and inserts are properly installed and sealed.
5. Fill the fluid reservoir with about 50 ml of clean water or a simulated urine fluid. The opening for the fluid is located in the upper part of the dog's body. It is advisable to use a syringe with a tube for easy filling.
6. The simulator is ready for use. You can practice the procedure of catheterization.
7. You can also perform perrectum examinations on the simulator using an anal insertion for this purpose.

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### Educational Features:

- Training to locate the urethral opening in female dogs
- Realistic simulation of resistance during catheter insertion
- Fluid flow response when catheter is correctly placed in the bladder
- Perrectum examination training

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## Training Instructions

1. Identify the vaginal entrance and locate the urethral opening – the simulator replicates realistic anatomical difficulty.
2. Gently insert a sterile catheter through the vestibule into the urethra.
3. Maintain the correct angle and depth of insertion – resistance will be felt.
4. Once the catheter is properly placed, fluid will begin to drain from the bladder.
5. After completing the urine collection procedure, slowly pull the catheter out of the system.

## Cleaning and Maintenance

- After training, empty the fluid reservoir.
- All silicone parts can be hand-washed with warm water.
- Do not use sharp tools or abrasive cleaners.
- Ensure all parts are fully dry before storage.
- Store at room temperature, away from heat sources and sunlight.

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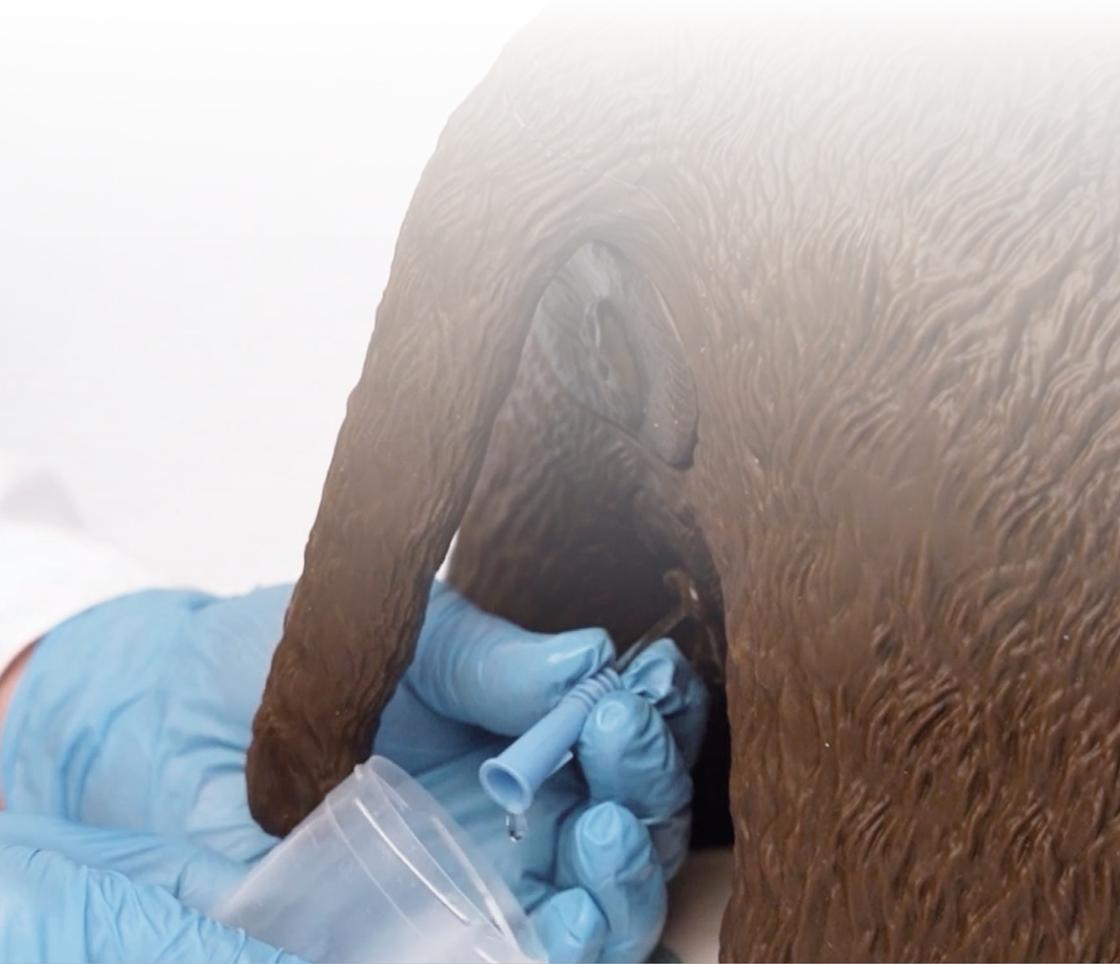
## Warnings and Safety

- The simulator is intended for educational use only.
- Do not use real medical products or biological fluids.
- Children may only use the simulator under adult or instructor supervision.

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**Erler-Zimmer GmbH & Co.KG**