„Lena Leg“ Bein mit Erkrankungen, inklusive Stativ
“Lena Leg“ Leg with medical conditions, including stand

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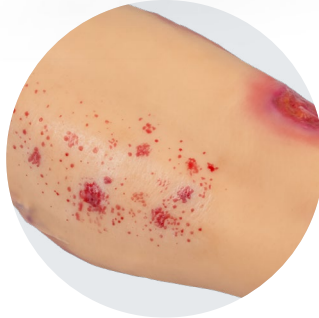
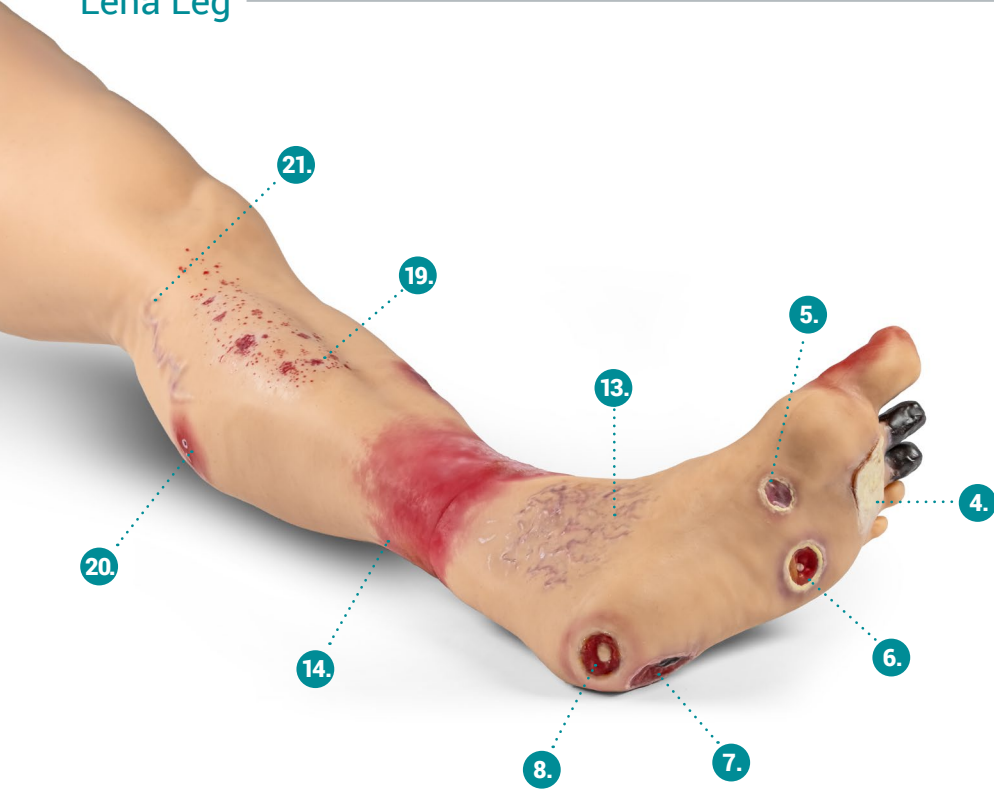
Leg with medical conditions

The Medical Leg Care simulator is designed for medical education and training in the visual assessment and differentiation of a wide range of lower limb pathologies and wound types. This simulator accurately replicates a human leg from the foot to mid-thigh. It supports diagnostic skills in wound care, vascular assessment, dermatology, and general medical training. The dimensions of the simulator are about 56 cm high, 23 cm wide and 20 cm deep.

The simulator shows such changes on the leg as: Ingrown Toenail, Toe necrosis, Fungal Thickened Toenails, Maceration, Neuropathic Pressure Injury Stage 3, Neuropathic Pressure Injury Stage 4, Heel Pressure Sores Stage 3, Heel Pressure Sores Stage 4, Cellulitis caused by an Ingrown Toenail, Atrophie Blanche, Stasis Dermatitis, Hemosiderin Staining, Corona Phlebectatica, Lipodermatosclerosis ("Champagne Bottle Leg"), Venous Ulcer, Healed Ulcer, Pyoderma Gangrenosum, Calciphylaxis, Vasculitis, Cellulitis caused by a Bite, Varicose Veins, Telangiectasia, Pitting Edema.

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Step 1:

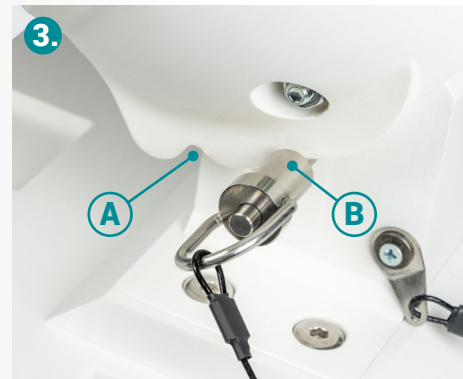
Position the clamp at the edge of the table. Tighten the screw until the clamp is securely fixed.

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Step 2:

Fully insert the locking pin into the designated opening on the model.

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Step 3:

Depending on the desired leg position, fully insert the locking pin into either opening A or B to secure the model.

EXPERTS IN MEDICAL EDUCATION

Erler-Zimmer GmbH & Co.KG

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T +49 (0)7841 / 67191-0 · F (0)7841 / 67191-99

info@erler-zimmer.de

www.erler-zimmer.de

Follow us!