Pneumothorax-Trainer

Pneumothorax Training Manikin

Anleitung / Manual

Art. Nr. / Item No. R10037

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About the Simulator

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This trainer is designed to assist in the teaching of procedures in relieving a tension pneumothorax. These instructions do not include medical procedures describing how to complete a pneumothorax reduction.

The Pneumothorax Simulator consists of a rigid upper torso with a simulated ribcage, covered with a foam liner, and then a lifelike vinyl skin. The simulator can be transported easily in its custom carrying bag.

The Pneumothorax Simulator is provided with an assembled foot-pump for easy inflation of the pleural space bladder. The replaceable bladder has two compartments and is reversible. That allows for usage of both sides of the pleural bladder.

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Loosen plastic cover secured with hook and loop fastening at the base of the manikin (See Figure 1).

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Turn cover 180° to access the interior of the manikin. (See Figures 2).

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Unplug the tubing connector from the valve attached to the pleural bladder (See Figure 3.)

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Unsnap the bladder from inside the shoulder area (See Figure 4.)

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Fold the double bladder so the used half is down (See Figure 5.)

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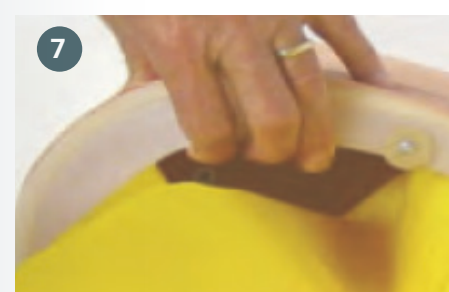
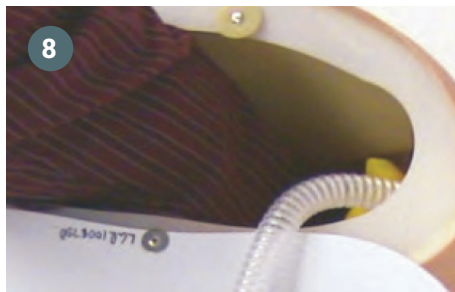
Slide the bladder into the torso (See Figure 6.)

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Plug connector half way into the valve of the top bladder (See Figure 8.)

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Turn the cover up into place, align hook and loop discs and press to secure (See Figure 9.)

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Each side of the cavity may be punctured many times before replacement is necessary.

NOTE:

It is not necessary to deflate pleural cavity after class session however, should you desire to do so follow directions 1 through 3 above.

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Cut away the old skin exposing the foam-covered ribcage (See Figure left).

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Loosen Velcro cover at base of manikin and turn cover 180° to access the interior of the manikin.

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Unplug the connector from the valve attached to the plural cavity and pull tubing through the hole in the side to the outside of the manikin.

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Powder the inside of the new skin and the outside foamed shoulder area of the manikin. (Use talcum powder or baby powder).

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Place the opening of the new skin over one shoulder about 2" down a little at a time alternating the shoulders. Once the skin is past the breast area it will begin to slide easier (See Figure 5).

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Pass tubing assembly connector through the hole to inside of manikin and reconnect to valve on pleural bladder (See Figure 6).

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Turn the bottom cover up into place. Align Velcro discs and press to secure.

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Wash with warm, soapy water after each use, rinse with clear water, and wipe dry with a soft cloth. Do not immerse.

Caution: *Manikins' soft flesh-colored parts are extremely porous and can be easily stained by ink from pens or printed material, lipstick, grease, and dirt. Do not let these items come in contact with manikin or stains will most surely occur and be difficult or impossible to remove. There is no warranty coverage for ink stains.*

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Erler-Zimmer Medical GmbH