Puncture Pete

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IV arms are a key training tool in skills labs. This model has been completely redesigned to ensure realistic haptics, an authentic appearance, and reliable functionality. Typical requirements from practical use have been taken into account in the design.

+ Quick and straightforward skin replacement

The durable yet pleasantly soft silicone skin can be replaced within seconds. This allows the skin to be removed and reattached quickly and efficiently, enabling timely exchange of veins when required.

+ Simple and secure vein installation

The veins remain securely in place when the skin is applied or removed. Integrated retention features within the vein channels ensure that the veins are consistently correctly aligned and immediately ready for use.

+ Reliable flow without extensive preparation

The engineered fluid system enables rapid air removal within less than 15 seconds. This ensures reliable backflow and supports a realistic training experience.

+ Realistic anatomy for practical training

The visible and palpable veins precisely correspond to the underlying cannulatable venous system. This allows users not only to identify and palpate the veins but also to perform targeted cannulation and achieve realistic blood return.

+ Different skin tones for realistic simulation

For maximum training realism, the injection arm is available in different skin tones. Standard options include light and dark variants. Custom adaptations are available on request to meet specific requirements.

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Place the arm on an absorbent pad and pull the skin back to the distributor cover.

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Open the distributor cover and then check that all tubes are properly connected.

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*Hang the blood bag on the stand and connect it to the **long tube** coming out of the shoulder area.*

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Fill the blood bag with the supplied artificial blood.

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*Open the shut-off valve of the **short tube** over a suitable collection container.*

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Then open the shut-off valve of the blood bag.

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*Open the shut-off valve on the **short tube** to allow the veins to fill with artificial blood and venting the system.*

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*Close the shut-off valve on the **short tube** after venting to begin the training.*

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*After training, drain all blood into the collection container through the **short tube**.*

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Fill the blood bag with distilled water and thoroughly flush the system.

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*Continue flushing the system until clear water flows from the **short tube** without any artificial blood residue.*

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Clean the puncture sites on the arm to remove artificial blood residue.

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Carefully remove the skin cover and leave it turned inside out.

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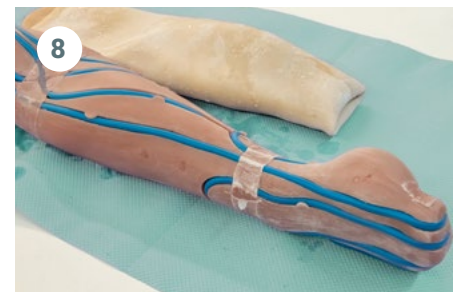
Rinse the inner surface of the skin thoroughly with water to remove the artificial blood.

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
Also clean the arm core and veins under running water.

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Allow the arm core and the skin, turned inside out, to dry completely.

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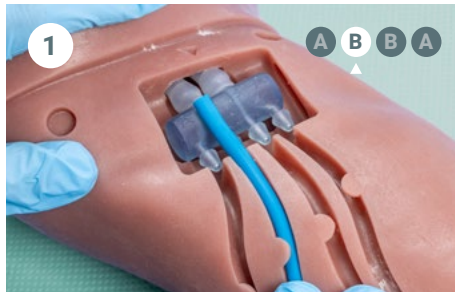Start at the left connector “B” and insert the vein into the channel below, ensuring a sufficient overhang.

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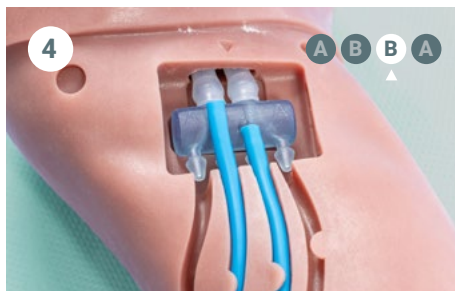
Insert the vein along the channel throughout the entire arm core and use tweezers to pull it beneath the tissue at concealed locations.

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Press the vein down into the channel at the crossing points so that the vein above can be easily inserted.

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
Continue along the channel to the outlet at the right connector “B” to close the circuit, leaving sufficient vein overhang.

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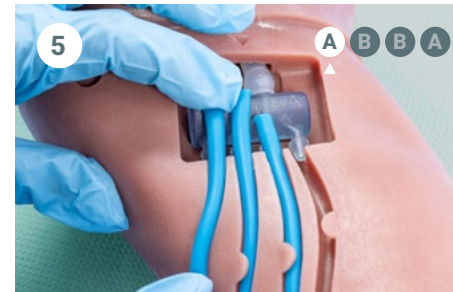
Repeat the procedure with the second vein at the left connector “A”, also leaving sufficient overhang.

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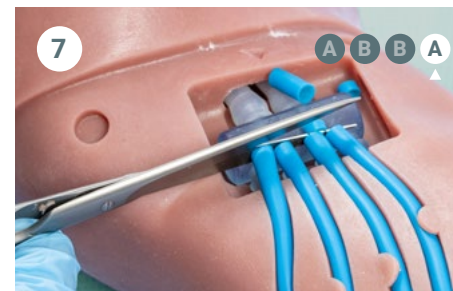
Ensure that the vein is routed under the nipple tabs at the relevant position.

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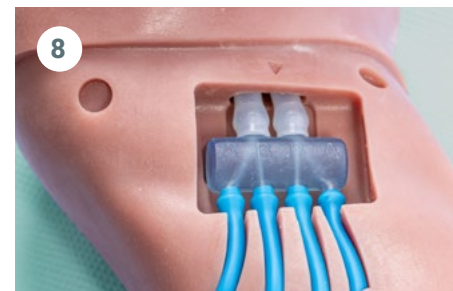Continue along this channel to the end at the right connector “A” and evenly trim the vein overhang behind the connector.

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Finally, attach the trimmed ends individually to all connectors to create a closed circulation system.

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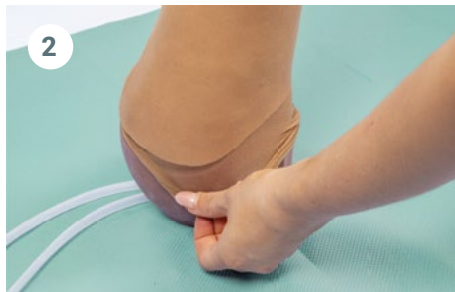
For skin removal, use a nylon stocking (not included) fitted over the arm starting from the hand end.

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Place the arm upright and pull the stocking to the end of the skin. Then fold it inward by approximately 2 cm.

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Now grip under the edge of the skin and remove it upward together with the stocking.

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Finally, peel the skin completely off the arm core.

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Slide the new skin over the arm core from the hand side, starting at the top and moving downward.

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Carefully slide the skin over the entire arm core, section by section.

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The supplied powder reduces friction for smoother movement of the skin.

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Pull the skin to the edge of the arm core until it is flush.

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The video guide makes replacing the skin quick and easy!

- + Use only 20 or 22 gauge needles (yellow or black).
- + Regularly inspect the venous tubing, skin, and distributor for any signs of damage, and replace components if necessary or when wear is observed.
- + When storing the arm or leaving it overnight, ensure that nothing is placed under the arm (including supply tubing), as this may create pressure points that can permanently damage the skin over time.
- + These arms are continuously being improved and further developed. Check for leaks, excessive wear, or any other issues that may indicate possible product improvements. Please share your experiences and feedback with us so that we can continue to optimize the product.
- + Using a nylon stocking (not included), placed carefully over the arm, can help make removal of the silicone skin easier by reducing frictional resistance.

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Silicone is durable but not indestructible. Do not overstretch the skin. Avoid contact with sharp-edged or pointed objects such as watches, jewelry, or similar items, as these may cause damage. Use only artificial blood that does not permanently stain the skin (Item No. 7024).

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BEFORE USING THIS PRODUCT, PLEASE READ THE TERMS AND CONDITIONS SET FORTH IN THIS LICENSE AGREEMENT. YOUR USE OF THIS PRODUCT SHALL CONSTITUTE ACKNOWLEDGMENT AND ACCEPTANCE OF THESE TERMS AND CONDITIONS.

If you do not agree to use this Product pursuant to the terms and conditions set out in this License Agreement, please contact Erler-Zimmer within ten days of receipt to return the unused Product for a full refund. The purchase of this Product conveys to you, the buyer, the non-transferable right to use the purchased Product for Licensed Use (see definition below) subject to the conditions set out in this License Agreement.

This Product may not be used for any purpose other than Licensed Use. Your right to use this Product for Licensed Use is subject to the following conditions and restrictions:

1. „Licensed Use“ means use for training purposes. „Licensed Use“ does not permit licensing others to use the Product or selling, distributing or otherwise providing the Product to any third party. You may provide the Product to individuals employed or enrolled with your organization.
2. Modification, reproduction based on Product design, and reverse-engineering of the Product in any way is strictly prohibited.

Your right to use the Product will terminate immediately if you fail to comply with these terms and conditions. Made under license from The Curators of the University of Missouri for technology developed at the University of Missouri-Columbia Shelden Simulation Center.

EXPERTS IN MEDICAL EDUCATION

Erler-Zimmer Medical GmbH

Hauptstrasse 27 · 77886 Lauf · Germany

T +49 7841 / 67191-0 · F +49 7841 / 67191-99

info@erler-zimmer.de

www.erler-zimmer.de

Follow us!