# Rex

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390 × 415 × 140 mm

### List of components:

1. Silicone body with embedded anatomical structures and a realistic paw
2. Replaceable genital insert
3. Artificial bladder
4. Tubing system for fluid flow

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## Product Description

The simulator is designed for educational and training purposes in the correct technique of catheterization of a male dog. It realistically replicates the anatomical structure of the male canine urogenital tract, including the penis, urethra, os penis, and bladder. In addition, the simulator is equipped with a realistic paw that allows users to practice bandaging.

The model presents the pelvic region of a medium-sized male dog in a lateral position, with one pelvic limb and tail movable to simulate clinical positioning during the procedure.

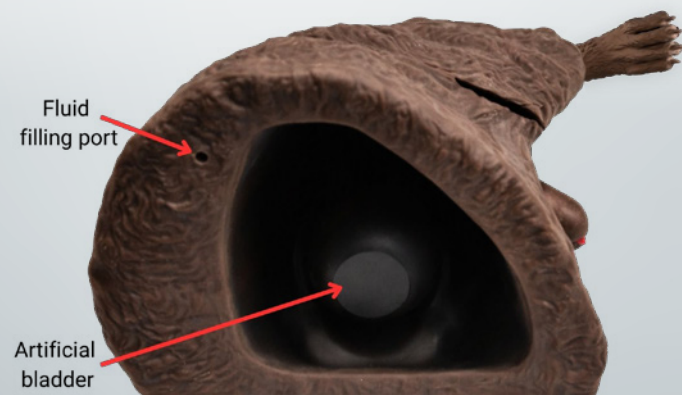
The simulator includes a replaceable genital insert with retractable silicone prepuce, allowing realistic training in catheter insertion through the penile urethra. A built-in fluid system enables urine flow simulation upon correct catheter placement.

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## Educational Features

- *Training in male dog urethral catheterization*
- *Realistic anatomical pathway including penile urethra and os penis*
- *Simulation of physiological resistance during catheter insertion*
- *Retractable prepuce enabling realistic access to the penis*
- *Fluid flow response when catheter is correctly placed in the bladder*
- *Movable tail and pelvic limb for proper positioning*
- *Realistic paw with toes for bandaging*

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## Preparation for Use

1. Place the simulator on a stable, flat surface.
2. Ensure the genital insert is properly installed and secured.
3. Check that the internal bladder (silicone reservoir) is correctly connected to the urethral system.
4. Filling the internal fluid reservoir:
  - Prepare a syringe filled with approximately 100 ml of clean water or simulated urine fluid.
  - Attach a catheter (6Fr, 180mm) to the syringe.
  - Insert the catheter into the designated fluid filling port.
  - Gently compress the internal bladder to remove air and create negative pressure.
  - Slowly press the syringe plunger while gradually releasing the bladder.
  - Repeat the process if necessary until water begins to flow out from the upper opening, indicating that the system is fully filled.
5. Verify all connections are tight and leak-free.
6. Position the tail and limb to simulate clinical catheterization positioning.
7. The simulator is ready for use.

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## Catheterization Instructions

1. *Retract the prepuce to expose the penis.*
2. *Identify the urethral opening at the tip of the penis.*
3. *Gently insert a lubricated catheter (6Fr, 450mm) into the urethra.*
4. *Advance the catheter slowly, following the curvature of the urethra.*
5. *Expect realistic resistance, especially at the level of the os penis and urethral bends.*
6. *Continue insertion until the catheter reaches the bladder.*
7. *Correct placement is confirmed by fluid flow from the catheter.*
8. *After completing the procedure, slowly withdraw the catheter.*

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## Cleaning and Maintenance

- *After training, empty the fluid reservoir completely.*
- *All silicone components should be cleaned with warm water.*
- *Do not use sharp tools, alcohol-based or abrasive cleaning agents.*
- *Allow all components to dry completely before storage.*
- *Store at room temperature, away from direct sunlight and heat sources.*

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### Warnings and Safety

- *The simulator is intended for educational use only.*
- *Do not use biological fluids or real patient materials.*
- *Use only appropriate training catheters (recommended 6 Fr)*
- *Improper use may damage internal structures.*
- *Use under supervision in training environments.*