Elektronisches Blutpumpmodul Bloody M.A.R.C. Electronic Bleeding Pump System Bloody M.A.R.C.

Bedienungsanleitung / User Manual

Art. Nr. / Item No. NW990

Herzlichen Glückwunsch Congratulations

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for purchasing the Electronic Bleeding Pump System
Bloody M.A.R.C.

Please follow the instructions in this manual to guarantee
the best functio-nality and allow you to enjoy the product
for a long time.

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Charge the Power Bank 4 and the
Pump Module using the correspond-
ing Power Supply for Power Bank 7.

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Open the Fluid Bladder 2 by turning
the cap clockwise in the direction of the
arrow.

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Fill the Fluid Bladder 2 with approxi-
mately 2.5 to 3 liters of artificial blood
through the opening.

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Connect the wound to the connection
to the tube of the backpack.

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Optionally, you can use the supplied
4-way stopcock bank Luer 11 for multiple
wounds.

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Plug the yellow connector into the yellow
socket to connect the Pump Module 3
to the Power Bank 4.

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After switching on the Power Bank 4
to position I, 4 LED lights indicate the
charge level.

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Turn the controls II and III on the
remote control 5 all the way to
the left.

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Turn on the Remote Control 5 by
pressing the On/Off Switch IV.

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*Set the Control Knob **II** to continuous operation mode (non-stop pulse) by turning it slightly.*

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*If you continue to turn the Control Knob **II**, pulse mode is activated continuously (approx. 50 to 220/min.).*

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*The flow rate can now be adjusted by turning the Control Knob **III** to the right.*

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*After finishing your training, remove the Fluid Bladder **2** and rinse it out with water.*

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*Disconnect the tube connection to the wound and switch the Power Bank **4** to the position **0**.*

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*Turn the Control Knobs **II** and **III** back and switch off the device using the On/Off Switch **IV**.*

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When the Fluid Bladder is full, the manifold must be closed to prevent fluid leakage.

The power bank must always be switched on before starting the control system. The yellow plug of the pump must not be plugged into the 24 V IN/OUT socket to avoid overvoltage.

Please refer to the instructions and guidelines for the safe use of the power bank in the included user manual.