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Art. Nr.: R19150B

Content

The CPR Rate Monitor (Rate Monitor, CPR Monitor, Monitor) is a patented feedback feature for Prestan Professional Manikins that consists of an LED light bank that responds to compressions (see the section on the CPR Rate Monitor). The CPR Rate Monitor (requires two (2) 'AA' batteries; not included) is pre-installed in the left shoulder of the Child manikin.

Cleaning

- The face and the body skin of the Prestan® Professional Manikin is made out of the same material and can be cleaned in the same manner.
- If the material becomes dirty, clean with a mixture of soap and water. If the area is extremely dirty, common household cleaners such as Formula 409®, Windex® or Vim® may be used. Non-bleach wipes such as Clorox® or Lysol® disinfecting wipes are convenient and work well. It is recommended that the use of these cleaners be done cautiously with minimum amount of abrasion to insure the life of the skin.
- A melamine foam eraser such as Mr. Clean® Magic Eraser® or Scotch-Brite™ Easy Erasing Pad may be used cautiously for stains.
- Cleaners containing wax, oil, or citrus (d-Limonene) are not recommended.
- Any cleaner not listed/recommended above may be tested in a non-visible area of the manikin. Open the manikin's torso chest and test the cleaner on the underside. Monitor for 24 hours. Should there be an undesirable reaction, the cleaner should not be used on the manikin.
- The face skin may also be sanitized with alcohol wipes or soaked in a solution of 1/4 cup bleach mixed with one gallon of water for 10 minutes. Between student uses, the instructor (or student) may clean the faces off with an alcohol wipe as an extra measure of safety.
- Don't forget to wipe the upper chest foam and skin in the same way in order to sanitize the inside of the manikin as well.
- Replacement skins for the face and torso may be purchased from your Prestan Authorized Distributor.

Storage

- Store the Prestan Professional Manikin(s) in the nylon carry bag(s).
- The manikin head should be removed for protection of the facial features, specifically the nose.
- For added protection, store the manikin inside the carry bag, and place the carry bag inside the original shipping box.
- Do not stack the manikins unless they are inside the original shipping box or other protective container.
- The nylon carry bag for the Prestan Professional Manikin(s) is not meant to store or transport belongings (i.e. books) in addition to the manikin(s) - please use the carry bag only for the intended manikin(s) to prevent damage to the bag.
- Store the manikin(s) in a cool, dry area, at temperatures between 50°-70° F and 50% relative humidity.
- For storage longer than 1 month, it is recommended that the batteries be removed from the CPR Rate Monitor (if the manikin has this feature).

CPR features

FACE-SHIELD/LUNG-BAG FOR THE VISIBLE CHEST RISE

- The Face-Shield/Lung-Bag combines the Face Shield and the Lung Bag into a single, t-shaped airway for the manikin. Air goes in through the Face-Shield opening to inflate the Lung- Bag, which provides the visible chest rise in response to administered CPR breaths.
- Face-Shield/Lung-Bags are sized specifically to fit each Prestan Professional Manikin (Adult/Child/Infant) and are not interchangeable among the different manikin sizes.
- The Face-Shield portion covers the mouth of the manikin. It stays in place by hooking the ends onto buttons located at each side of the head under the ears.
- The Lung-Bag portion is placed on the manikin chest (under the torso skin) and fills with air from administered CPR breaths. This provides a visual chest rise for the student.
- Face-Shield/Lung-Bags are a consumable item and should be changed depending on how they are used:
 - If breaths are being administered by mouth, either directly or via rescue mask adaptor, the Face- Shield/ Lung-Bag should be changed between classes due to accumulation of bacteria-causing moisture inside the neck and bag. This change is advised for both sanitary and performance purposes.
 - If breaths are being administered by a manual resuscitator (BVM/Ambu bag), there is no need to change the Face-Shield/Lung-Bag.
- When a manikin is being shared by more than one student, it is recommended that each student be provided with their own sanitary and disposable mouth protector:
 - If direct mouth contact is being used to give CPR breaths, students should be supplied with their own Face Shield (sold separately and disposable). The fitted Face Shield can be placed over the already-installed Face-Shield/Lung-Bag on the mouth of the manikin when it is that student's turn to give breaths. Generic, non-fitted face shields may also be used, but may slide around on the mouth and possibly cover the mouth opening if not held in place.
 - If rescue masks are being used to give CPR breaths, students should be supplied with their own Rescue Mask Adaptor (sold separately and disposable). The adaptors can be inserted into any standard mask when it is that student's turn to give breaths.

CPR CLICKER

- The Clicker is the round disk in the center of the manikin's back.
- The Clicker works with compressions to provide an audible "click" when compression depth reaches 2" (the compression is able to continue to a 2.4" depth).
- The Clicker sends a signal to the CPR Rate Monitor (if the manikin has this feature) to allow it to provide feedback on compression speed (see CPR Rate Monitor – Guide)

CPR Rate Monitor Guide

The CPR Training Manikin Rate Monitor has a unique way of monitoring both the depth of compressions and the rate of compressions.

At the first compression all four of the lights in the upper left shoulder will come on, indicating that the lights are working and the batteries are good. (Figure 1)

If the lights do not light, make sure that the depth of compression is deep enough (you will hear a clicking sound). The lights will not work if the student is not compressing the manikin correctly.

If the depth of compression is correct, and the lights do not come on, replace the two AA alkaline batteries in the battery compartment located in the back of the manikin in the left shoulder.

Once a student begins chest compressions, the amber light and the green lights will go out and, from zero to less than 60 compressions per minute, the red light will be on. (Figure 2)

As the student increases the rate of compressions and reaches 60 compressions per minute, the red light will go off and the amber light will come on until the student reaches 80 compressions per minute. (Figure 3)

As the student increases the rate of compressions and reaches 80 compressions per minute, the amber light will go off and first green light will come on until the student reaches 100 compressions per minute. (Figure 4)

When the student reaches 100 compressions per minute, the second green light comes on, indicating that the proper rate as been attained. (Figure 5)

If the student slows down, one of the green lights will go out indicating that the students needs to increase the rate of compressions.

Should compressions exceed 120 per minute, a yellow light will flash rapidly indicating the recommended cap has been reached. (Figure 6)

Always try to maintain the recommended speed between 100 and 120 compressions per minute as indicated by the 2 green lights.

If the student's depth of compression is not correct, at any time, all the lights will go off and the red light will blink.

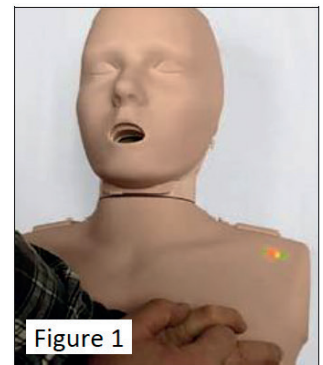
Figure 1

Figure 2; <60 cpm

Figure 3; 60-80 cpm

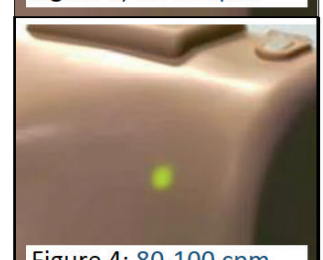
Figure 4; 80-100 cpm

Figure 5; 100-120 cpm

Figure 6; >120 cpm

Face-Shield/Lung-Bag Installation

1. Unlatch the torso of the Prestan Professional Manikin by popping the 'tabs' located at the top of the shoulders (Figure 1). Carefully lift the torso frame open by the neck. **Note: Do not pull on tabs (past popping the latch). Doing so may break the tabs or stretch out the frame over time, impairing the latching function.**

Figure 1

2. Attach the head to the torso by placing the neck of the head onto the two spindles on the neck portion

Figure 2

3. Fold the Lung-Bag portion of the Face-Shield/Lung-Bag (Figures 3a-3d):

- The Face-Shield/Lung-Bag has four 'dotted lines' printed on one side of the bag (along with an oval)
- Using the four dotted lines, fold the Face-Shield/Lung-Bag into fourths, following the diagram below

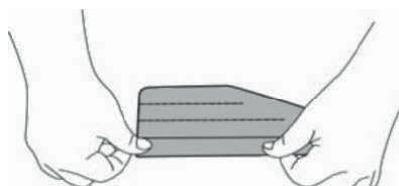
Figure 3a

Figure 3b

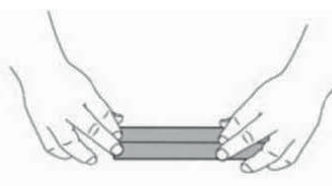
Figure 3c

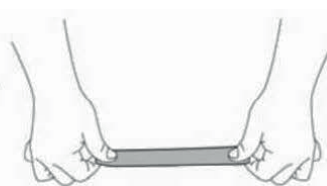
Figure 3d

4. Tilt the head of the manikin (Figure 4).

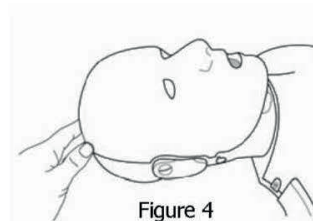
Figure 4

5. Insert the lung bag, folded end, into the mouth of the manikin (Figure 5).


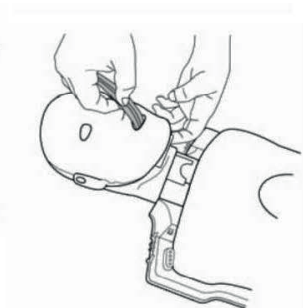
Figure 5

6. Push the lung bag into the mouth until the bag exits through the neck (Figure 6).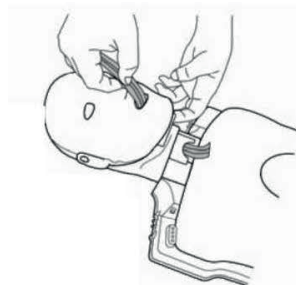

7. Grasp the lung bag and pull it through the neck (Figure 7)

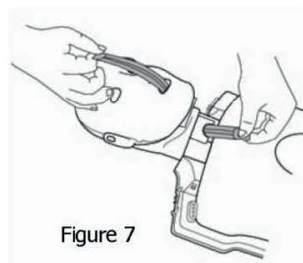
Figure 7

8. Attach the Face-Shield tabs to the 'buttons' located on the side of the manikin head (Figure 8).

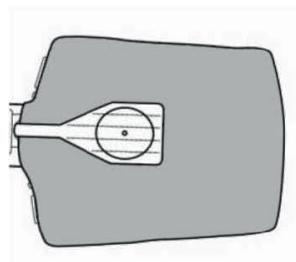
Figure 8

9. Lightly pull down on the Lung-Bag to help open the mouth hole on the Face-Shield.

10. Then position the Lung-Bag on top of the compression plate (the hard plastic circular piece found on top of the torso foam core) with the circular imprint on the Lung-Bag aligning with the compression plate (Figure 9).

Figure 9

11. With proper positioning, the neck of the Face-Shield/Lung-Bag will have slack. Make sure it is not twisted.

12. Close the torso and secure the latch by pushing on the shoulder tabs until they snap into place.

Troubleshooting

CPR Rate Monitor

Symptom	Possible issue	Solutions
Cannot find CPR Rate Monitor (Monitor) – no lights in the shoulder area	You have purchased a Prestan Professional Child Manikin without the CPR Rate Monitor	If you were expecting a monitored manikin, check with the distributor where you purchased the manikin to insure you received what you paid for. Note: CPR Rate Monitors may be purchased separately from your Prestan Authorized Distributor
CPR Rate Monitor (Monitor) lights are not working properly To help understand how the Monitor works, please note that the Monitor lights are powered by a small current generated by the Clicker Please follow the troubleshooting tips at right in the order they are listed for best results	- Chest compressions are not deep enough to signal Monitor - Batteries are not installed, need replacing or are not inserted correctly in the CPR Rate Monitor - The Clicker-Monitor connection is broken - Clicker may be clogged with plastic dust, interrupting the signal to the Monitor - Either the Clicker or the Monitor needs to be replaced	- Compress the chest until the click is audible from the Clicker at 2"-2.4" - Install or replace batteries into Monitor (back of manikin, left shoulder area) - requires 2-AA (1.5V) batteries - On the manikin's back, remove the long skinny Wire Cover between the Clicker and Monitor – squeeze connectors to disconnect the wires then reconnect to insure Clicker-Monitor communications - Remove Clicker and blow out dust – then replace Clicker and reconnect with Monitor - Exchange Clickers and Monitors with working manikins (if available) to determine whether the Clicker or the Monitor needs to be replaced*
CPR Rate Monitor (Monitor) lights are all on	Batteries are running low	Replace batteries
CPR Rate Monitor (Monitor) red light is blinking	This is normal after replacing batteries or manikin is paused - may blink up to 16 times to indicate ready status	Commence or resume compressions - blinking will stop and feedback rate signals will begin

* Contact your Prestan Authorized Distributor to either purchase the replacement part or receive it at no charge if the manikin is within Prestan's 3-year warranty period and is a qualifying warranty repair.

Troubleshooting

Torso and Face-Shield/Lung-Bag

Symptom	Possible issue	Solutions
Cannot hear a click when doing compressions	•Hand position and/or compression angle may be incorrect •Compressions may not be deep enough to activate the clicker •Clicker may be broken	•Confirm correct hand placement and compress straight down from directly above the manikin •Compress between 2 and 2.4 inches depth •Replace Clicker in back of manikin*
Chest will not rise when breaths are given	•Head is not tilted back enough to open airway •Face Shield is not attached •Lung Bag may be damaged or torn (Note: there are holes in the Child Lung Bag necks by design to allow deflation) •Lung Bag is not lying flat or properly positioned over the compression plate and/or neck of bag is twisted •Lung Bag has accumulated moisture in the neck, which may cause the sides to stick together and prevent air passage	•Push on forehead and/or lift at chin to tilt head back enough to open airway •Attach Face Shield at “buttons” on the side of the manikin’s head below ears •Blow into the Face-Shield/Lung-Bag while it is outside the manikin to make sure it inflates properly – if not, discard and use another Face-Shield/Lung-Bag •Revisit instructions for installing a Face-Shield/Lung-Bag earlier in this manual • A new Face-Shield/Lung-Bag should be installed after each class, or more often if needed due to excessive moisture buildup
CPR Rate Monitor (Monitor) lights are all on Manikin torso will not stay closed Manikin torso opens when breaths are given	•Tabbed latches at shoulders may not be securely closed •Chest frame may be stretched from improper opening of torso •Tabbed latches at shoulders are broken from improper opening of torso	•Close tabs at shoulders – a snap will be heard when shoulder tabs are properly latched •Open torso and attempt to reverse stretched out frame by pushing it in the opposite direction •Torso Frame needs to be replaced*

Torso

Symptom	Possible issue	Solutions
Rattling in chest	Compression Sleeve may be broken	•Confirm correct hand placement and compress straight down from directly above the manikin •Compress between 2 and 2.4 inches depth •Replace Clicker in back of manikin*
Chest Plate is broken	Typically results from a combination of incorrect (not centered) hand placement, compression angle not straight down from directly above and/or overzealous compression force	Replace chest plate with RPP-CCOM-1 (compression assembly includes chest plate, spring, sleeve and clicker)*

* Contact your Prestan Authorized Distributor to either purchase the replacement part or receive it at no charge if the manikin is within Prestan’s 3-year warranty period and is a qualifying warranty repair.

Consumables

	Sanitary Airways for R19150 Pack of 50 pieces Item no.: R19150A
	Face shields for R19150 Pack of 50 pieces Item no.: R19150B

*Experts in medical education
since 1950*

Erler-Zimmer GmbH & Co.KG

Hauptstraße 27

D-77886 Lauf

Tel.: +49 (0)7841 / 6003-0

Fax: +49 (0)7841 / 6003-20

E-Mail: info@erler-zimmer.de

Available 24/7:

www.erler-zimmer.de

Follow us!