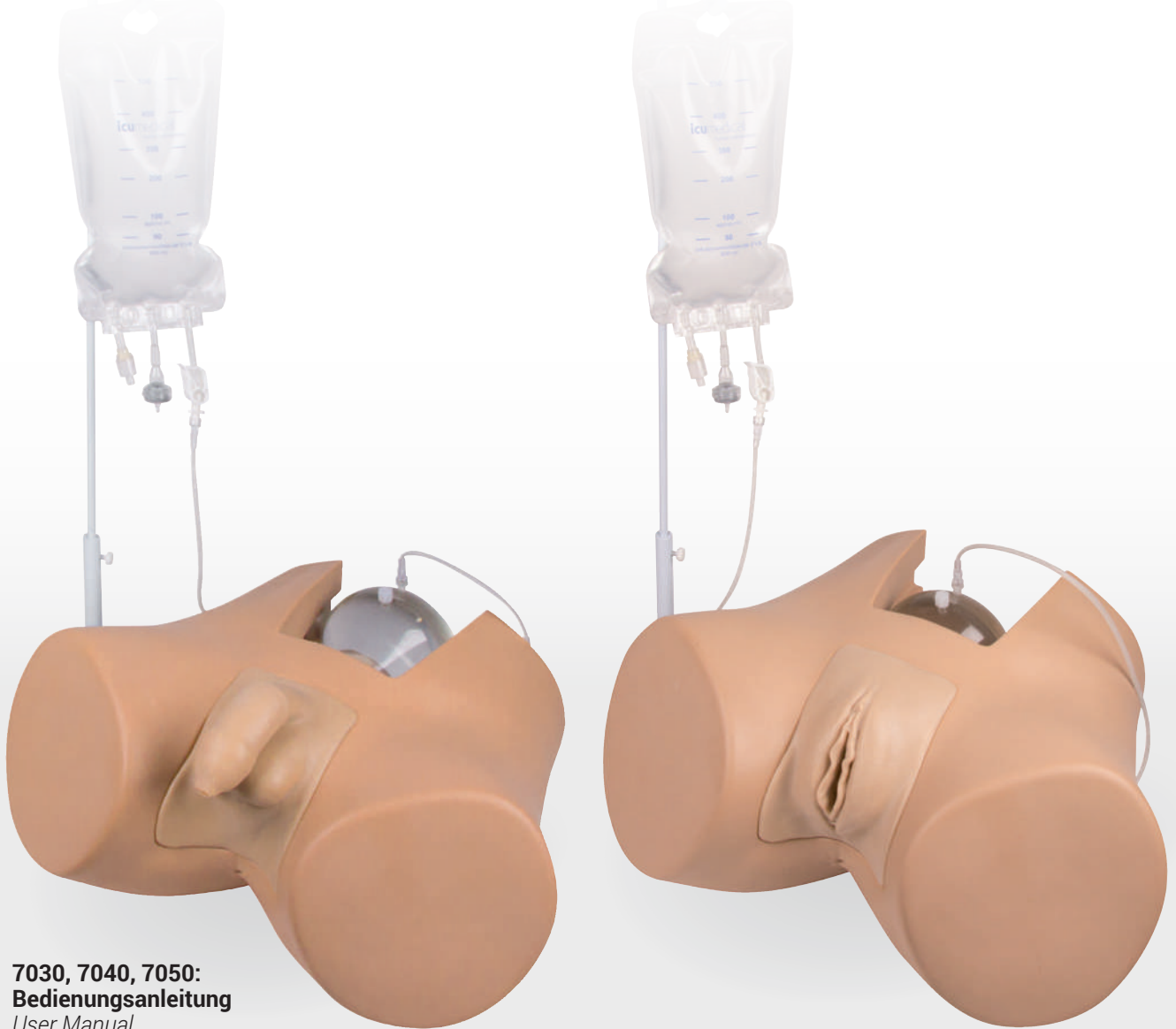
Katheterisierungsmodell

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General Instructions for Use

Florence and Henri are simulated patients. Treat them as you would do with real patients and they will stay in good condition for a long time.

Just as with real patients, Florence and Henri should not be handled or scratched with long fingernails or come in contact with harmful objects, including sharp rings. You should never pull or tug on the anatomy, including the penis and labia. This is especially true for the delicate foreskin.

Florence and Henri are professional training tools, not toys. Do not leave the models with students unattended. Florence and Henri should be used only for their intended purpose, training students in urinary bladder catheterization.

Follow the instructions of this manual. Appropriate use and care will ensure that you can use the products for your lessons for many years without any problems.

If something does not work as expected, please refer to the „troubleshooting“ part of this manual for quick guidance. If you are still experiencing difficulties, please contact your distributor or Erler-Zimmer's customer service.

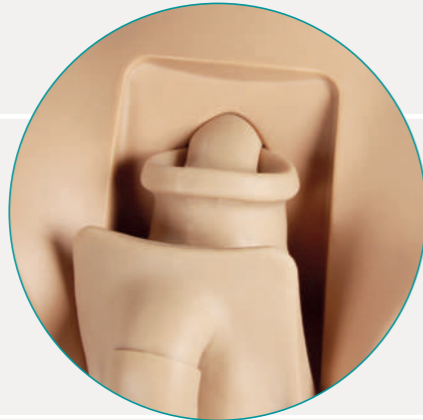
Florence and Henri can be disinfected and catheterized like real patients. You may use normal disinfectants and catheters. However, avoid the use of disinfectants that contain colors (e.g. Braunol). Lubricants with anesthetics can be used without any concerns (e.g. Instillagel).

To guarantee a proper function, a sufficient amount of lubricant must be used at all time. For easy cleaning, we suggest the use of water-based lubricants. It is also recommended to lubricate the entire urethral tract before use. Inject lubricant into the urethra using a syringe and fully insert a catheter to make sure the lubricant is spread through the entire canal. Prepping the model in this way will result in the most life-like catheterization experience.

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
Inserting the genitalia


1. To place the insert into the body, compress the inner part of the insert and place it into the hole on the front side of the body.

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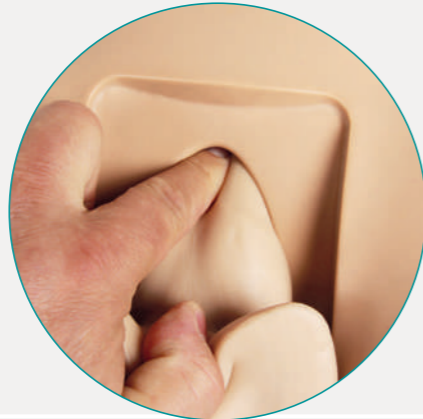
2. Push the insert into the hole using your fingers. Do NOT use your fingernails!

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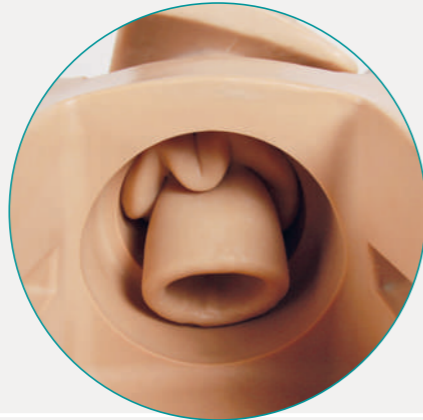3. Grab the insert from inside of the body and pull it into the final position. Make sure to grab the insert securely and not on the edge.

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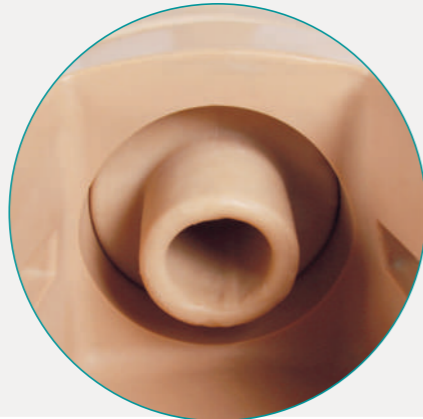4. After the inner part of the genital is in place, push the outer edges of the insert into place to make sure the edges match the body shape.

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Attaching the foreskin

Apply water based lubricant over the entire glans and about 1 cm behind it. Hold the foreskin at the end and spread it slightly.

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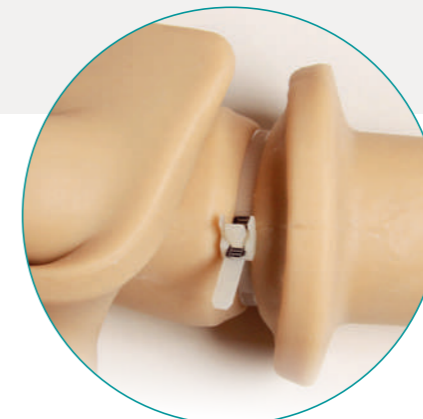
Pull the foreskin over the penis until it is covered completely. It should be possible to retract the foreskin. If this does not work, apply more lubricant.

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Attaching the prostate restriction

The model allows for simulation of urethral stricture caused by prostate cancer or BPH. To simulate this, a supplied plastic strap can be fixed along a special groove around the inner part of the insert (see image). Tighten the strap to the black marks. The strap can be released by pressing the release mechanism.

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Filling the bladder

Unscrew the bladder. You will find two valves inside the screw cap. Fill the bladder with water (distilled water recommended). The bladder has two openings, each with a Luer connection. Both are closed with a Luer cap, when closing the bladder, take care to first insert the cross-shape valve and second the lamella valve. Both should point into the bladder to guarantee proper function.

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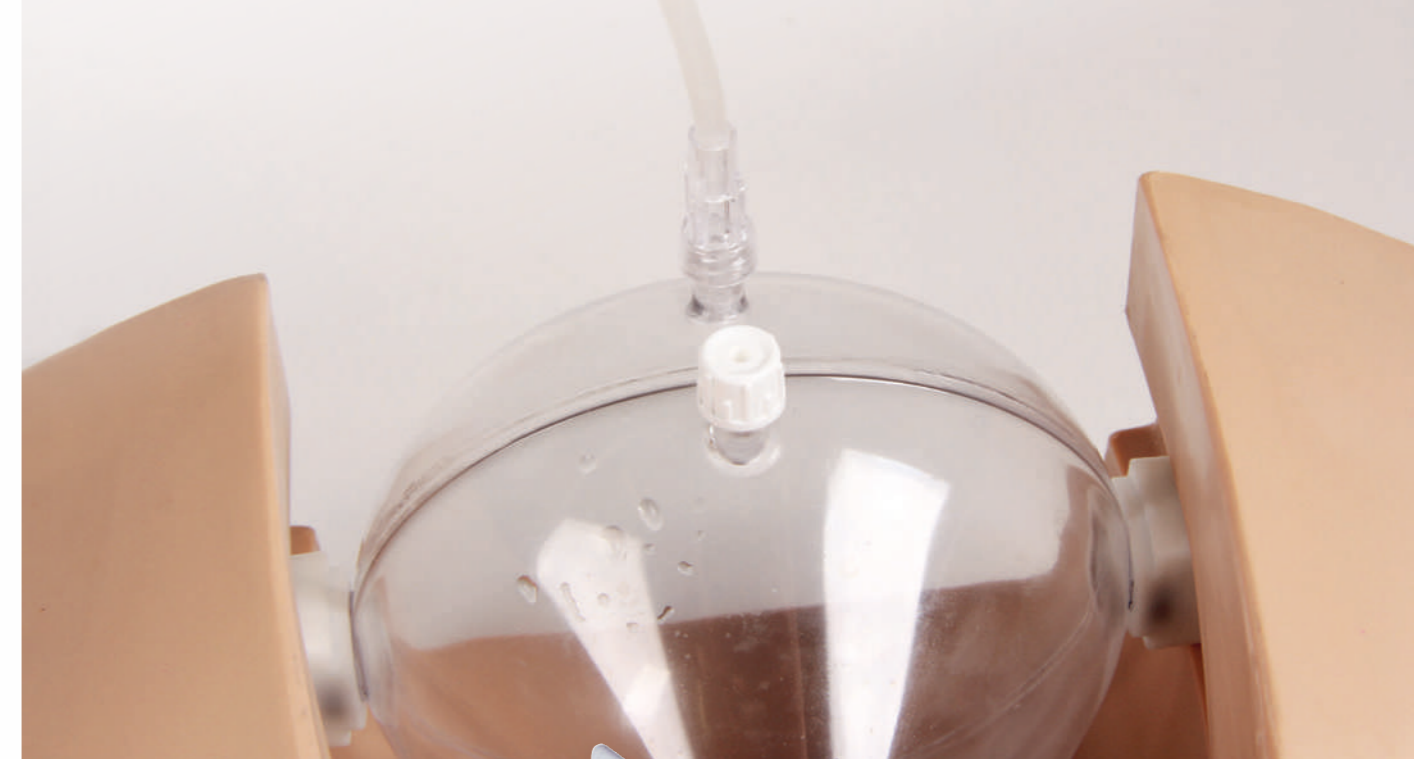
Inserting the bladder

Turn the white guide blocks on the sides of the bladder into horizontal position for use with the female insert and in vertical position for use with the male insert.

Place the bladder in the body and while making sure that the white guide blocks align with slits on the left and right. The shape of the slits combined with the vertical or horizontal position of the guide blocks will guarantee the correct insertion depth of the bladder for both male or female inserts.

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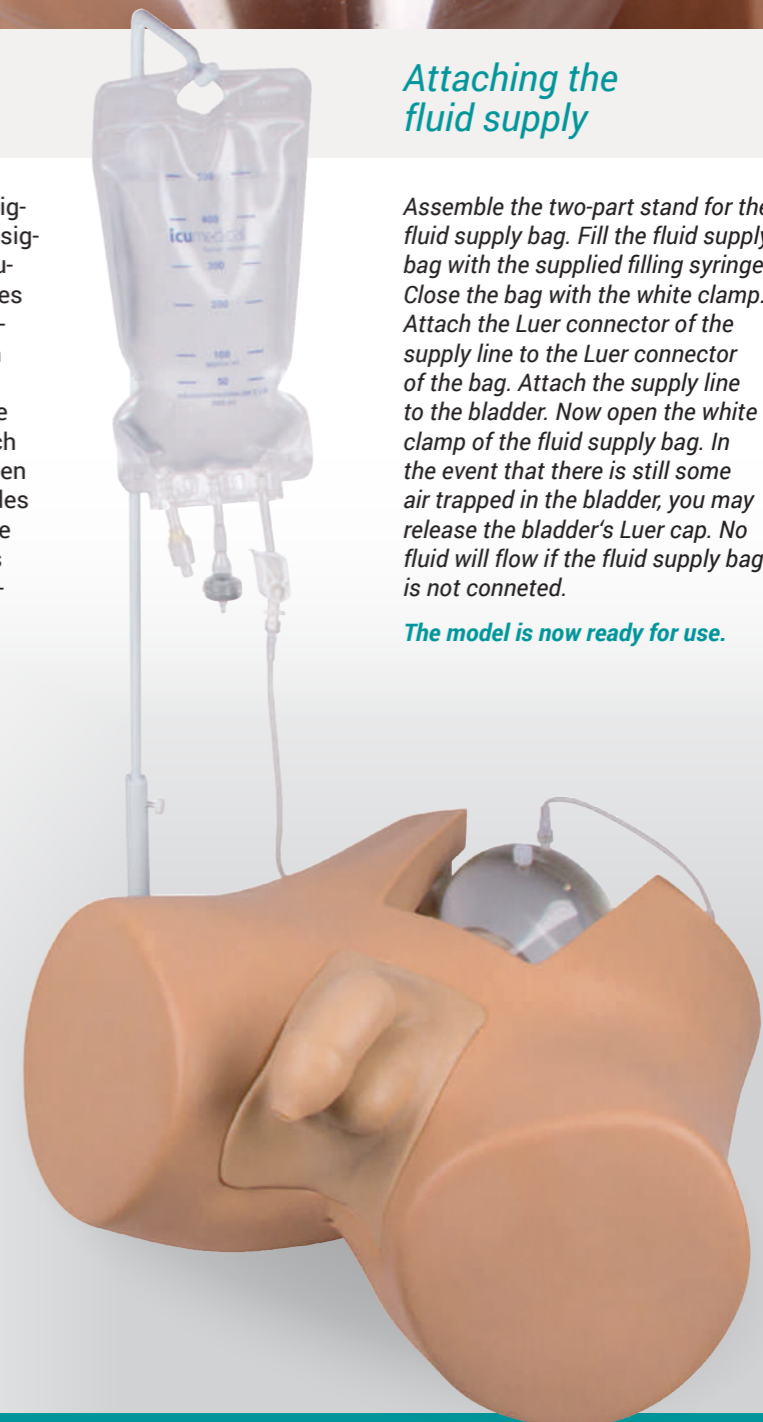
Attaching the fluid supply

Assemble the two-part stand for the fluid supply bag. Fill the fluid supply bag with the supplied filling syringe. Close the bag with the white clamp. Attach the Luer connector of the supply line to the Luer connector of the bag. Attach the supply line to the bladder. Now open the white clamp of the fluid supply bag. In the event that there is still some air trapped in the bladder, you may release the bladder's Luer cap. No fluid will flow if the fluid supply bag is not connected.

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Performing catheterization

Treat the model carefully and use gloves, just as you would do with a real patient. When retracting the foreskin, pull back delicately. Rough handling or long fingernails can damage the inserts.

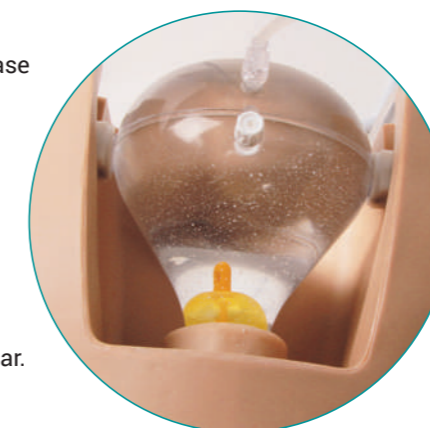
The sterilization process can be performed just as it would on a real patient. Take care NOT to use disinfectants containing colors.

It's best to use the same normal lubricants that you would use on patients. Always make sure to use sufficient amounts of lubricant.

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Florence and Henri allow the use of all common catheters. Use the same sizes you would typically use with patients. The resistance within the urethra simulates real anatomical resistance and the catheter must be passed using correct manipulation techniques.

Special valves in the bladder create little to no resistance. This is intentional and realistically represents the very subtle sensation of passing a catheter into the human bladder. After successfully entering the bladder, fluid will flow through the catheter. The transparent bladder also allows for valuable visualization of Foley catheter placement. The balloon can be inflated and placed in view of the teacher and trainee for excellent training experience.

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After the training

To maintain the long-term function of the model it should be cleaned and prepared for storage directly after the training.

Empty the fluid supply bag, the supply line and the bladder. Make sure all parts have fully dried before storage.

Remove the insert from the body. Make sure to pull on the middle section, not on the edges of the inserts (see images).

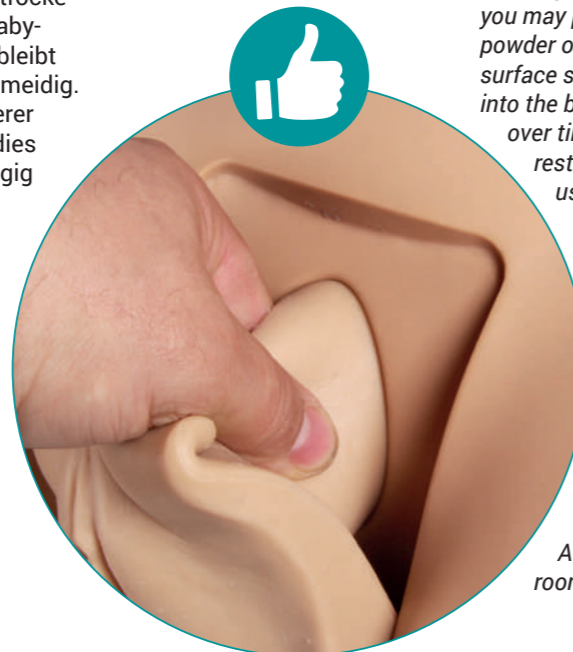
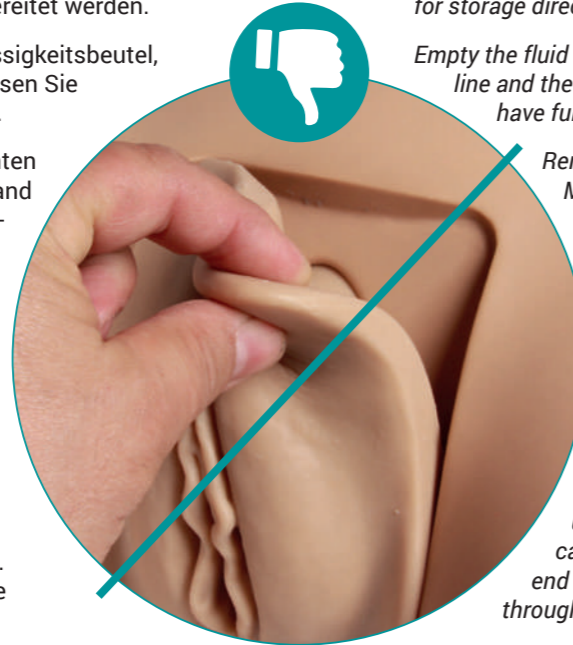
To remove all lubricants, wash all inserts and the foreskin you have used carefully with lukewarm water.

Also flush the urethra with water. You may either use a syringe to do this or you can fill water through the inner end if the insert. Fluid will flow through the urethra flushing it.

Dry the inserts carefully after washing. After drying the inserts you may powder them with baby powder or talcum powder, making the surface soft, smooth and easy to insert into the body. If the surface gets sticky over time, it's always an option to restore the original surface feel using powder.

Store the inserts either in the body or separately. Take care to store the inserts so that nothing lies on top of them and they are not compressed or pinched. This may lead to permanent deformation.

Store the model dry and without direct sunlight. A dark and air-conditioned room is ideal for storage.

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Foreskin will not retract back properly.

- not enough lubricant.
- Glans not covered completely with lubricant
 - > Remove foreskin and apply lubricant onto glans and about 1 cm behind it. Make sure the whole surface is covered.

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No fluids flows despite the catheter having entered the bladder.

- Bladder filled, but fluid supply is not properly connected.
 - > Connect fluid bag, open white clamp.
- White clamp on fluid is closed
 - > Take catheter out, flush it and try again.

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Bladder cannot be placed in correct position.

- Guide blocks are in the wrong position for the anatomical insert that is in use.
 - > Remove bladder, turn blocks 90°, insert again.

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Ref. no.	Description
7030	Florence and Henri Catheterization model, dual sex
7040	Henri Catheterization model, male
7040A	Male Genital Insert with foreskin
7040B	Replacement foreskin
7045	Lower body with bladder
7045A	Fluid supply bag
7045B	Supply line
7030S	Stand
7050	Florence Catheterization model, female
7050A	Female genital insert

*Experts in medical education
since 1950*

Erlor-Zimmer GmbH & Co.KG

Hauptstraße 27

D-77886 Lauf

Tel.: +49 (0)7841 / 6003-0

Fax: +49 (0)7841 / 6003-20

E-Mail: info@erler-zimmer.de

Available 24/7:

www.erler-zimmer.de

Follow us!     